

## STATE OF OKLAHOMA BOARD OF TESTS FOR ALCOHOL AND DRUG INFLUENCE

Dr. Jarrad Wagner, Ph.D. F-ABFT Chairman

Post Office Box 36307 Oklahoma City, Oklahoma 73136-2307 Phone: (405) 425-2460 Fax: (405) 425-2490 www.bot.ok.gov

Joshua Smith Director

#### **ATTESTATION**

I, the undersigned <u>Joshua Smith</u> (Director/Records Custodian), attest under oath pursuant to 12 O.S. §2902 (11)(a) this record was made, at or near the time of the occurrence of the matters set forth by or from information transmitted by a person having knowledge of those matters; was kept in the course of the regularly conducted business activity of the Board; and was made pursuant to the regularly conducted activity. The duplicate record provided is listed below:

1) The attached "Service Overview" is a true and accurate copy of maintenance performed on the Intoxilyzer 8000, serial number 80-003393, in compliance with the agency's Maintenance Policy and Procedures.

This packet consists of 13 pages.

signed \_\_\_\_\_

Name of Position: Director/Records Custodian

Date of Attestation: 05/19/2022

### Service Overview



3600 N. Martin Luther King Bldg #9 Oklahoma City, OK 73111 405-425-2460

BENCH CHECK DATE: 07-09-2019

**SERIAL:** 80-003393

### **DOCUMENTATION WITHIN SERVICE PACKET:**

- BOT Certificate of Calibration and Operation
- Bench Check
- Bench Check Technician's Report
- BOT Dry Gas Certificate of Analysis
- Mock Subject Test
- BOT Certificates of Analysis on Certified Solutions (4)



## STATE OF OKLAHOMA BOARD OF TESTS FOR ALCOHOL AND DRUG INFLUENCE

Administrative Office at:
Board of Tests for Alcohol and Drug Influence
Post Office Box 36307
Oklahoma City, Oklahoma 73136-2307

### Certificate of Calibration and Operation

This is to certify that the calibration of INTOXILYZER 8000, serial number 80-003393, manufactured by CMI Inc., a subsidiary of MPD, Inc. of Owensboro, Kentucky, was tested and found to be in compliance with National Highway Traffic Safety Administration Standard for Devices to Measure Breath Alcohol (F.R., No. 179 48705-48710 Sept. 17, 1993) and with International Recommendation OIML R 126 Edition 1998 (E). Calibration solutions are traceable to NIST standard material 1828.

In addition the operational procedures and functions of this Intoxilyzer have also been tested and found to be in compliance with the Oklahoma Board of Tests for Alcohol and Drug Influence - Rules and Regulations OAC Title 40: 25-1-2 and 30-1-3 and is hereby certified for use as an evidential breath testing device within this state.

CERTIFICATION TYPE	DATE
Manufacturer Calibration	08-26-08
Board of Tests Bench Check	11-14-08
Board of Tests Bench Check	03-23-10
Manufacturer Calibration	10-06-10
Board of Tests Bench Check	10-28-10
Board of Tests Bench Check	10-06-11
Manufacturer Calibration	06-28-13
Board of Tests Bench Check	07-09-13
Board of Tests Bench Check	06-23-15
Board of Tests Bench Check	05-17-17
Board of Tests Bench Check	09-24-18
Board of Tests Bench Check	03-25-19
Board of Tests Bench Check	07-09-19



Presentation of this form certifies that the Intoxilyzer, SN: 80-003393, functioned properly at the time of the breath test and hereby certifies the breath test result as valid.

Date:

Re: Request for Log of Test on Intoxilyzer 8000

Name:

#### Dear Sir:

The Log of Test information from a breath test conducted on a Intoxilyzer 8000 is printed in Section 2 of the Affidavit and is no longer retained as a separate form as with the Intoxilyzer 5000. The 8000 uses a nitrogen/ethanol dry gas for the external control test. The dry gas cylinder has the capacity to perform approximately 325 tests and does not require maintenance until its expiration date or can no longer provide the required pressure to perform a control test. The cylinder has an expiration date of two years from date filled at the manufacturer and is programmed in the 8000 upon installation, both dates are printed in Section 2 of the Affidavit. The Intoxilyzer is programmed to disable itself when the date of expiration is reached or the pressure monitor indicates 25 lbs psi.

Due to the reliability and stability of dry gas, maintenance on the 8000 is not required every 30 days or 25 tests, therefore maintenance is no longer performed by officers in the field. The Board of Tests for Alcohol and Drug Influence certifies each Intoxilyzer 8000 for field use by performing a bench check. The bench check is designed to check the operation and procedures programmed in the 8000 to ensure it is functioning properly for use in Oklahoma. A bench check is performed every time a new dry gas cylinder is installed or the instrument has returned from the manufacturer for repair. Once the instrument passes the bench check, all that is required to set it up for field use is to plug it in and turn it on and does not require a permit to set up. The Board of Tests has the ability to monitor each instrument once it is set up in the field.

Each Intoxilyzer has a database of valid permit numbers that it checks prior to allowing a breath test to be performed. The database is updated on a regular basis, if a permit number is invalid that officer will not be able to conduct a breath test on the 8000. If the 8000 prints an affidavit the breath test operator listed in Section 2 had a valid permit at the time of the test and the breath test is valid.

If you have any further questions please feel free to call me.

Sincerely.

J. Robert Blakeburn (OBA11059) State Director of Tests for Alcohol and Drug Influence

O. pohrt Blokelun

# STATE OF OKLAHOMA - BOARD OF TESTS FOR ALCOHOL AND DRUG INFLUENCE CMI INTOXILYZER BENCH CHECK PROCEDURE REPORT

NCH CHECK DATE 07/09/2019	12:52:1	SAN	INSTRUMENT LOCATION 80-003393					
RUMENT TYPE ntoxilyzer	INSTRUM	ENT MODEL		INSTRUMENT SERIAL NUMBER 80-003393				
rcoxilyzei	1-0		ESULTS	00.00	80-003393			
Diagnostic PASS Mouth Alcoho PASS RFI Test	ol Test							
PASS		Du	plicate Breath	Sampling Re	sults			
Abort Tes PASS	st	Value	Test 1	Test 2	Control Test			
Improper Samp PASS	le Test	.02	0.018	0.017	0.017			
Interferent Dete	ect Test	.04	0.038	0.037	0.037			
No Sample Give	en Test	.10	0.099	0.099	0.100			
PASS		.20	0.199	0.198	0.198			
Insufficient Sam PASS Air Blank Contamir PASS	rougen Danke Jali		PA	SS				
Range Exceede PASS	ed Test							
		CERTIFYING	TECHNICIAN					
swear under penalty he Intoxliyzer Bench				f Oklahoma.				
MITH, JOSHUA				PERMIT#	016837			



3600 N. Martin Luther King Bldg #9 Oklahoma City, OK 73111 405-425-2460

### **BENCH CHECK TECHNICIAN'S REPORT**

itoxilyzer Model	5	Serial #	
000		80-003393	
SAS CANISTER			
OT#	, E	XP Date	
00919080A2	# 004	03/05/202	
REFERENCE		19/33/5/5	
O: #h 0400	Simulator		O. # 0400
Guth 2100	Guth 2100 Seria	Guth 2100	Guth 2100
DR3591	DR3753	DR3594	DR3754
	Concen	tration	
.02	.04	.10	.20
18020	17410	18070	17340
10020	Manufactu	VA.E.1.7	17540
01-09-2018	12-06-2017	02-26-2018	10-09-2017
	Expiration		18,83,0740
01-09-2020	12-06-2019	02-26-2020	10-09-2019
	Solution Com	mission Date	
07-09-19	07-09-19	07-09-19	07-09-19
01-01-11	01-01-1-1	0.0111	01-04-1
	NCH CHECK ]CYLINDER REPLA ING ☑ROUTINE N		
TROUBLESHOOT			
OMMENTS	x valve on so	ince Block.	
OMMENTS	k valve on so	LRCE BLOCK.	

Printed Name

g/210L

0.000

0.079

0.000

0.079

0.000

0.080

0.000

0.080

0.000

0.079

0.000

0.080

0.000

0.081

0.000

0.079

0.000

0.080

0.000

0.079

0.000

0.0796

0.0006

0.8783

SN 80-002591

10:05

Time

10:05

10:05

10:06

10:06

10:07

10:07

10:08

10:08

10:09

10:09

10:09

10:10

10:10

10:11

10:11

10:11

10:12

10:12

10:13

10:13

10:14

ALPHA INSTRUMENT

Model 8000

03/25/2019

Test

Air Blank

Cal Check

Air Blank

Std Deu

Cal Check Stats Average

Rel Std Deu(%)

Intoxilyzer - Alcohol Analyzer

Certificate	e of Analysis - Dry Gas				
03/25/19	,080				
Date of Analysis	Labelled target value (g/210L)				
# 004	,000				
Cylinder#	Average test result				



In accordance with BOT Policy and Procedure Statement BRT-2.1.0, Maintenance, the above referenced dry gas cylinder is suitable for use as an external reference with the Intoxilyzer 8000.

0 · Jacksonville, IL 62651-0790

21/-245-2165 · rax: 21/-243-7634 · www.ilmoproducts.com

BRT Form 5.0

### Certificate of Analysis

Certificate ID:

BOT Technician Name and Signature

11722

Part #:

BAC105L080T

Cylinder Size:

105L

Lot Number:

00919080A2

Expiration:

3/5/2021

pecvis 03-05-201

 $0.080\,$  BAC (For the calibration of instruments used to determine breath alcohol concentration)

Contents:

105 Liters @ 1000 psig 70°F (21°C)

Analytical

Reported

Accuracy

Analytical

Component:

Concentrational

(U, k=2):

Method:

Ethanol. Nitrogen '

208 ppm Balance +/-0.002 BAC(G/210L) NDIR

[5.2 ppm]

Distributed by:

CMI Inc.

316 East Ninth Stre Owensboro, KY 421 Phone 866-835-069 www.alcoholtest.co

rator's Signature

\*NIST Traceable Reference Material Cylinder No. CC274523 / job No. 09160306 Certified 362.2 µmol/mol Ethanol in Nitrogen

Store in dry area, away from sources of heat, ignition and direct sunlight. Do not allow storage area to exceed 52 °C (125 °F).

Spegialty Gas Lab Tech

Accreditation #61895

The calibration results within this certificate were obtained using equipment and standards capable of producing analytical results traceable to NIST, and apply only to the items contained on this certificate. ILMO Products Company makes no warranty or representation as to the suitability of the use of any information provided for any particular purpose. The information use is at the sole discretion and risk of the user. Liability shall be limited to established replacement cost of this material or service.

# OFFICER'S AFFIDAVIT AND NOTICE OF REVOCATION/DISQUALIFICATION

. .

										Section		
RREST DATE 07/09/2019		REST TIME		COUNTY 55	#	63		ONE				
REST LOCATION NONE				CITY   NONE				COUNTY				
JBJECT NAME					DAT	E OF BIRTH			CE HEIGHT	WEIGHT		
	CT TEST				0	8/09/1			V 510			
odress 3600 MARTIN	LUTHER K	ING A	CITY	LAHOMA (	CITY		STA		ZIP COE 731			
RIVER LICENSE #				EXPIRATION	DATE	STATE	CLASS	RE	STRICTIONS	ENDORSEMEN		
T083454871 HICLE MAKE		MODEL		06/30	)/201	OK TAG#	A	STATE	CMV	M		
NONE		NONE				NOEN		OK		N		
the above date, time, and loca nway, street, tumpike, public pa sence of alcohol and/or other in scribe driving behavior or circu MOCK SUBJECT	arking lot, or other public p ntoxicating substances as p imstances):	lace, or upon a private prohibited by law.	road, street, alley	or lane which provide								
scribe person's condition (odor,	actions etc.):											
NONE	, actions, etc.).											
NOINE												
		THE PERSON V	VAS INFORMED	OF THE IMPLIED CO	NSENT TEST	REQUEST						
EATH-ALCOHOL ANALYSIS I	RECORD, REPORT OF TH	E PERSON NAMED	IN SECTION 1 AE	BOVE, and LOG OF T	EST AND MA	NTENANCE RE	CORD.			Section		
UIPMENT - 8000			NUMBER - 003393	SPECIALIST BOARD	OF T	בפייפ						
ANDARD		180-	TARGET VA		PERMIT NU							
ory			0.08	0	1683			Tours over	NOED EVEIN	-0		
nufacturer LMO			LOT# 0091	.9080A2		NDER INSTALLI			INDER EXPIRE $05/202$			
SERVATION START TEST		MOUTHPIECE		nute (or greater	<u> </u>				,			
335 07	/09/2019	G		l was observed					TE S	T		
Test	g/210L	Time	t	to the time of te	est admini	stration.			OF THE O	ATE		
			т 1 ' '	. 1 117	1 .	1 1 1 7		2	Z+.+./\$	1.4.4		
Air Blank	0.00 0.00	13:51 13:52		stered said I			1		ON			
Subject Test Breath Volume	1.601 LITERS	13:52	_	ance with the		_	- 1	H	70	7-15		
Air Blank	0.00	13:52	pro	ocedure of th	e Board	of Tests.	.		**	** 6		
Mait	0.00	13:54						18				
Air Blank Subject Test	0.00 0.00	13:54 13:55						•				
Breath Volume	1.726 LITERS	13.33	SIGNATURE	OF OPERATOR					180			
Air Blank	0.00	13:55	NAME									
Cal Check	0.08	13:56	SMITH,	JOSHUA					Test Re			
Air Blank	0.00	13:56	BADGE #	· · · · · · · · · · · · · · · · · · ·	PERMIT #	04.600	_		BrAC g/	210L		
			000000 AGENCY	00435	10000	01683	/		0.0	0.0		
				OF TESTS								
rective thirty (30) DAYs is or more, if you refused or ferating a CMV or CDL holder ety must receive your written a approved hearing requestision, Department of Public Sine of not more than Five Hunson's license, permit, or other	failed the state's test(s).  OR a test result of .04  request for an administ  t form is available at ht  Safety, P.O. Box 11415, (  dred Dollars (\$500.00),  r evidence of driving priv	In addition, your cor or more while opera ative hearing on the tps://www.ok.gov/o DKC, OK 73136. It s or by both such fine ilege is in the custod	nmercial driving pring a commercial Department applys/documents/hall be a misdemand imprisonmer y of a law enforcial	privilege in this state al motor vehicle. To proved form, within ReqHearing.pdf. So neanor punishable by ht, for any person to be ment officer or the	will be disquappeal the dr appeal the dr fifteen (15) of ubmit your he y imprisonme apply for a re Department.	alified if you ref iver's license re lays from the d aring request f nt for not less t newal or a repl	used or failed evocation/dis ate you rece orm in perso han seven (7 acement lice	d the state's qualification ived notice nat 3600 N days nor inse to oper	test(s)  In, the Department  In	r disqualification by mail to Legal 6) months, or by hicle while the		
rving of notice: 1 te served: 07/09				ntaining the Notic						ve.		
accordance with Title 12										Section		
e: 07/09/201		•		•		re of arrestir	0 0	13 (100 a)				
MITH, JOSI	HUA		,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,,	AGENCY BOARD	OF T	ESTS						
	AGENCY ADDRESS INTER-AGE	ENCY		DOMIL	, OF 1	ZIP CODE	) 0 PHON		52460	11		
HER WITNESSES:										Section		
NAME			TITLE	ADDRESS			JF	PHONE				
NAME			TITLE	ADDRESS				PHONE				
2			1	ADDRESS			-	TIONE				

### 9/13

# OFFICER'S AFFIDAVIT AND NOTICE OF REVOCATION/DISQUALIFICATION

				_							Section
07/09/2019		ARREST TIME		COUNTY 55	#	CIT		NO	NE		
ARREST LOCATION NONE					CITY	)NE			C	OKLAH	Δ 
SUBJECT NAME					D	ATE O	BIRTH	SE	X RA	ACE HEIGH	IT WEIGHT
ADDRESS	ECT TEST		CITY	7.1.01.7		08/	09/196	STATE		W 51	DDE
3600 MARTI	N LUTHER	KING A	OKL	AHOMA ( EXPIRATION			STATE  CL	OK ASS		73 ESTRICTIONS	133
T083454871		IMODE		06/30	)/201		OK Z	A			M
NONE		NONE	}			TAG N	OEN		STATE		PLACARD N
On the above date, time, and le highway, street, turnpike, public influence of alcohol and/or othe (Describe driving behavior or city MOCK SUBJECT	c parking lot, or other put er intoxicating substance: ircumstances):	blic place, or upon a private s as prohibited by law.	e road, street, alley or	lane which provide							
Describe person's condition (or	dor, actions, etc.):										
NONE											
			WAS INFORMED OF				`				
BREATH-ALCOHOL ANALYS EQUIPMENT	IS RECORD, REPORT C		NUMBER	VE, and LOG OF T	EST AND M	IAINTE	NANCE RECORE	). 			Section 2
I-8000		80-	-003393	BOARD	OF S						
STANDARD Dry			0.080		PERMIT I		:R				
MANUFACTURER			LOT#		DATE CY	LINDE	RINSTALLED	(		LINDER EXPIR	
	ST DATE	MOUTHPIECE		080A2 ute (or greater	<u> </u>	·	/2019		03/	05/20	Z T
1335 0	7/09/201	9 G		was observed				11			
Test	g/210L	Time		the time of to	•		~			OF THE	STATE
			T 1	1 '17	1	A 1	1 1 7		2	/	1 Kal
Air Blank Subject Test	0.00 0.00	13:51 13:52		tered said I						TO W	
Breath Volume				ice with the			-	g	Ы		
Air Blank Wait	0.00	13:52 13:54	proc	edure of th	e Boar	a oi	lests.	i		**	**   5
Air Blank	0.00	13:54							/3		
Subject Test	0.00	13:55	SIGNATURE O	E OPERATOR				-		190	07
Breath Volume Air Blank	1.726 LITER 0.00	RS 13:55	SIGNATURE	OLIVATOR							
Cal Check	0.08	13:55	NAME							Test R	esult
Air Blank	0.00	13:56	SMITH, BADGE#	JOSHUA	PERMIT#	ı				BrAC g	
			0000000	0435			16837			0	0.0
			BOARD O	F TESTS						<u> </u>	
EFFECTIVE THIRTY (30) DA days or more, if you refused of (operating a CMV or CDL hol Safety must receive your writ The approved hearing requ Division, Department of Publi a fine of not more than Five I- person's license, permit, or of	or failed the state's test der) OR a test result of ten request for an admest form is available to Safety, P.O. Box 114 lundred Dollars (\$500.)	c(s). In addition, your control of .04 or more while operatinistrative hearing on the at https://www.ok.gov/c15, OKC, OK 73136. It soon, or by both such fine	mmercial driving priving a commercial ne Department apprides/documents/Reshall be a misdemea and imprisonment,	vilege in this state notor vehicle. To coved form, within a qHearing.pdf. So anor punishable by for any person to	will be disq appeal the fifteen (15 ubmit your l imprisonn apply for a	ualifie driver ) days hearing nent fo renew	d if you refused of a license revocal from the date you grequest form in a not less than so	or failed the tion/disque ou receive ou person a even (7) d	he state's alification ed notice at 3600 N days nor	s test(s) n, the Departr of revocation I. MLK Ave. o more than six	or disqualification. or by mail to Legal or (6) months, or by
SERVING OF NOTICE:	I personally hand	-delivered a copy of	the above conta	nining the Notic	e of Revo	ocatio	on to the perso	on name	ed in Se	ection 1 abo	ove.
Date served: 07/0		Signature of officer							Badge		
In accordance with Title	12 O.S. Section 42	26, "I state under <b>pe</b>	nalty of perjury	under the law	s of Okla	homa	that the fore	going is	true ar	nd correct."	Section 4
Date: 07/09/20	19 Place (location	on when signed):			_ Signa	ture c	of arresting of	ficer:			
SMITH, JO BADGE	SHUA			BOARD	OF		STS				
0000000435	AGENCY ADDRESS INTER-A	GENCY					ZIP CODE 00000	"4(	)542	52460	) "



590 NORTH 67th STREET 9 HARRISBURG, PA 17111-4511 9 TELEPHONE: 71

09-26-2018 Date of Analysis 18020

Labelled target value (g/210L)

0.0204

Average test result (g/210L)

Certificate of Analysis - Wet Bath

In accordance with BRT - 2.1.0, Maintenance, the above referenced simulator solution is suitable for use as an external reference in maintenance of the intoxilyzer 8000.

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 18020 Alcohol Reference Solution for Simulator were analyzed by chromatography on January 11, 2018, using a Perkin Elmer Gas gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.0238% (w/vol) ethyl alcohol. The expiration date for this lot number is January 9, 2020 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.020 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Operator's Signature

RIPHR INSTRUMENT

Model 8000

09/26/2018

Test

Air Blank

Cal Check

Air Blank

Cal Check

Air Blank Cal Check

Air Blank Cal Check

Air Blank

Cal Check

Air Blank

Cal Check

Air Blank

Cal Check

Air Blank

Cal Check

air Blank

al Check

ir Blank

'al Check

IIr Blank

Anstage Std Deu

al Check Stats

Rel Std Deu(%)

Intoxilyzer - Alcohol Analyzer

g/210L

.0.000

0.020

0.000

0.020

0.000

0.020

0.000

0.020

0.000

0.020

0.000

0.020

0.000

0.021

0.000

0.021

0.000

0.021

0.000

0.021

0.000

0.0204

0.0005 2.5312 SN 80-002591

15:36

Time

15:36

15-37

15:37

15:38

15:39

15:39

15:40

15:41

15:41

15:42

15:43

15:44

15:44

15:45

15:46

15:46

15:47

15:47

15:48

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN08031603 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.



Certificate of Analysis - Wet Bath 1/13 09-26-18 Labelled target value (g/210L) Date of Analysis 0.0396

BOT Technician Name and Signature



CERTIFICATE OF

In accordance with BRT - 2.1.0, Maintenance, the above referenced simulator solution is As National State of the Association of the Associa

Certified Alcohol Reference Solution for Simulator

ALPHA INSTRUMENT Intoxilyzer - Alcohol Analyzer Model 8000 SN 80-002591 15:54

09/26/2018 Test g/210L Time Air Blank 0.000 15:54 Cal Check 0.040 15:55 Air Blank 0.000 15:56 Cal Check 0.039 15:56 Air Blank 0.000 15:57 Cal Check 0.039 15:58 15:58 Rir Blank 0.000 Cal Check 0.040 15:59 Air Blank 0.000 15:59 Cal Check Air Blank Cal Check 0.040 16:01 Air Blank 0.000 16:02 Cal Check 0.039 16:03 Air Blank 0.000 16:03 Cal Check 0.040 16:04 Air Blank 0.000 16:04 Cal Check 0.040 16:05 Air Blank 0.000 16:06 Cal Check 0.039 16:06 Air Blank 0.000 Cal Check Stats Average Std Dev

1.3040

Rel Std Deu(2)

Random Samples of Lot Number Alcohol Reference Solution for Simulator were analyzed by chromatography on December 7, 2017, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to 0.0481% (w/vol) ethyl alcohol. The expiration date for this lot number is December 6, 2019 at 11:59 PM.

When used in a calibrated Simulator, operating at +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.040 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.


Operator's Signature

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN12181501 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.

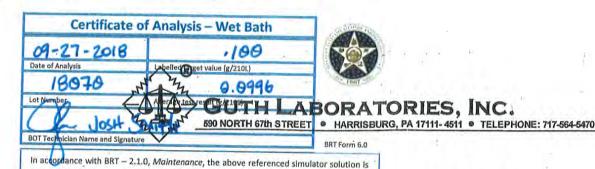

suitable for use as an external reference in maintenance of the Intoxilyzer 8000.

CERTIFICATE

OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 18070 of

Alcohol Reference Solution for Simulator were analyzed by
gas chromatography on February 28, 2018, using a Perkin Elmer Gas
Chromatograph Autosystem XL S/N: 610N9030209, and found to contain
0.1219% (w/vol) ethyl alcohol. The expiration date for this lot
number is February 26, 2020 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Operator's Signature

ALPHA INSTRUMENT

Model 8000

09/27/2018

Test

Air Blank

Cal Check

Air Blank

Average Std Deu

Cal Check Stats

Rel Std Deu(%)

Intoxilyzer - Alcohol Analyzer

0,000

0.100

0.000

0.100

0.000

0.099

0.000

0.100

0.000

0.100

0.000

0.100

0.000

0.099

0.000

0.099

0.000

0.099

0.000

0.100

0.000

0.0996

0.0005

SN 80-002591

08:28

08:28

08:29

08:29

08:30

08:31

08:31

08:32

08:33

08:33

08:34

08:34

08:35

08:36

08:36

08:37

08:38

08:38

08:39

08:39

08:40

08:41

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN04271602 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.

**BOT Technician Name and Signature** 

ALPHA INSTRUMENT

Model 8000

09/27/2018

Air Blank

Cal Check

Pir Blank

Cal Check

air Blank

Cal Check

Rir Blank

al Check

ir Blank

al Check

lir Blank

Std Deu

lal Check Stats Ruerage

Rel Std Deu(%)

Intoxilyzer - Alcohol Analyzer

g/210L

0.000

0.196

0.000

0.196

0.000

0.197

0.000

0.197

0.000

0.197

0.000

0.197

0.000

0.197

0.000

0.197

0.000

0.197

0.000

0.197

0.000

0.1968

0.0004

SN 80-002591

15:27

Tine

15:27

15:28

15:29

15:29

15:30

15:31

15:31

15:32

15:33

15:33

15:34

15:35

15:35

15:36

15:36

15:37

15:38

15:38

15:39

15:40

GUTH LABORATORIES, INC.

HARRISBURG, PA 17111-4511 TELEPHONE: 717-564-5470

RT Form 6.0

In accordance with BRT – 2.1.0, Maintenance, the above referenced simulator solution is suitable for use as an external reference in maintenance of the intoxilyzer 8000.

#### CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 17340 of
Alcohol Reference Solution for Simulator were analyzed by
gas chromatography on October 10, 2017, using a Perkin Elmer Gas
Chromatograph Autosystem XL S/N: 610N9030209, and found to contain
0.2410% (w/vol) ethyl alcohol. The expiration date for this lot
number is October 9, 2019 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.200 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Operator's signature

Ted L. Pauley, President GUTH LABORATORIES, INC.

· NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN08101505 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.