



STATE OF OKLAHOMA
BOARD OF TESTS FOR ALCOHOL AND DRUG INFLUENCE

Post Office Box 36307

Oklahoma City, Oklahoma 73136-2307
Phone: (405) 425-2460 Fax: (405) 425-2490

www.bot.ok.gov

Dr. Jarrad Wagner, Ph.D. F-ABFT
Chairman


Joshua Smith
Director

ATTESTATION

I, the undersigned Joshua Smith (Director/Records Custodian), attest under oath pursuant to 12 O.S. §2902 (11)(a) this record was made, at or near the time of the occurrence of the matters set forth by or from information transmitted by a person having knowledge of those matters; was kept in the course of the regularly conducted business activity of the Board; and was made pursuant to the regularly conducted activity. The duplicate record provided is listed below:

- 1) The attached "Service Overview" is a true and accurate copy of maintenance performed on the Intoxilyzer 8000, serial number 80-003394, in compliance with the agency's Maintenance Policy and Procedures.

This packet consists of 17 pages.

Signed 

Name of Position: Director/Records Custodian

Date of Attestation: 05/2/2022

Service Overview

Board of Tests for Alcohol and Drug Influence
*Enhancing public safety by ensuring the accuracy and scientific reliability
of blood and breath testing*



3600 N. Martin Luther King Bldg #9
Oklahoma City, OK 73111
405-425-2460

BENCH CHECK DATE: 05/02/2022

SERIAL: 80-003394

DOCUMENTATION WITHIN SERVICE PACKET:

- BOT Certificate of Maintenance, Calibration and Operation
- Bench Check
- BOT Dry Gas Certificate of Analysis
- Ilmo Specialty Gases Certificate of Analysis
- Mock Subject Test
- BOT Certificate of Analysis on Certified Solutions (4)
- Guth Laboratories Certificate of Analysis (4)



**STATE OF OKLAHOMA
BOARD OF TESTS FOR ALCOHOL AND DRUG INFLUENCE**

Post Office Box 36307
Oklahoma City, Oklahoma 73136-2307
Phone: (405) 425-2460 Fax: (405) 425-2490
www.bot.ok.gov

CERTIFICATE of MAINTENANCE, CALIBRATION, and OPERATION

This certifies the **Intoxilyzer 8000** referenced below, manufactured by CMI, Inc., a subsidiary of MPD, Inc. of Owensboro, Kentucky, is maintained by the Board of Tests for Alcohol and Drug Influence in accordance with 40 O.A.C. 30-1-3. The instrument reported below is hereby deemed to be in compliance with Board rules via linearity accuracy examination and operational/functional procedure evaluation also referred to as a "Bench Check". The Intoxilyzer 8000 appears on the National Highway Traffic Safety Administration's Conforming Products List of Evidential Breath Alcohol Measurement Devices (82 FR 50940 *et seq*) as an approved device. In accordance with 40 O.A.C. 25-1-2, this instrument is approved and certified for use in the State of Oklahoma as an automated evidential breath-alcohol measurement device by the Board of Tests pursuant to 47 O.S. § 759.

All analyses conducted by this instrument are considered precise, valid, and in accordance with 40 O.A.C. 30-1-3(b) Operating Procedure(s). A printed breath test operator affidavit containing the seal of the Board by this instrument confirms a calibration check successfully verified the instrument's calibration at the time of test. The printed breath test affidavit also serves as proof that safeguards in place to protect the breath samples, including but not limited to Ambient Air Interference, Radio Frequency Interference, Residual Mouth Alcohol, Deficient Samples, etc. were not triggered and the test is a true and accurate scientific measurement of the person's alcohol concentration.


- Maintenance was performed on the Intoxilyzer 8000, Serial No. 80-003394 on 05/02/2022 in accordance with 40 O.A.C. 30-1-3 and internal BOT maintenance policy and procedures.
- A dry gas canister appearing on the National Highway Traffic Safety Administration's Conforming Products List of Calibrating Units for Breath Alcohol Testers (77 FR 64588 *et seq*) and approved in accordance with 40 O.A.C. 25-1-3 and BOT maintenance policy was installed.
 - Manufacturer: ILMO Unit: 208 ppm / 105 L
 - Cylinder Lot No. - Canister No.: 35521080A3-011
 - BOT analysis average test result: 0.081
 - Cylinder expiration date: 03/05/2024
- In accordance with 40 O.A.C. 25-1-2.1, the approved reference standards and equipment were used:

Simulator Model			
Guth 2100	Guth 2100	Guth 2100	Guth 2100
Serial #			
DR3685	DR3694	DR3594	DR3754
Concentration			
.02	.04	.10	.20
LOT #			
21120	21070	21190	20510
Manufactured Date			
4/7/2021	3/2/2021	6/9/2021	12/3/2020
Expiration Date			
4/7/2023	3/1/2023	6/8/2023	12/3/2022
Solution Commission Date			
04/29/2022	04/29/2022	04/29/2022	04/29/2022

Completed by BOT Personnel:


Signature/Name Christopher Pape

Breath-Alcohol Testing Program Administrator
Title

Certified by:

Signature Name Joshua Smith

State Director
Title

STATE OF OKLAHOMA - BOARD OF TESTS FOR ALCOHOL AND DRUG INFLUENCE CMI INTOXILYZER BENCH CHECK PROCEDURE REPORT

BENCH CHECK DATE 05/02/2022	TIME PROCEDURE BEGAN 09:52:2	INSTRUMENT LOCATION 80-003394
INSTRUMENT TYPE Intoxilyzer	INSTRUMENT MODEL I-8000	INSTRUMENT SERIAL NUMBER 80-003394

TEST RESULTS

Diagnostics
PASS

Mouth Alcohol Test
PASS

RFI Test
PASS

Abort Test
PASS

Improper Sample Test
PASS

Interferent Detect Test
PASS

No Sample Given Test
PASS

Insufficient Sample Test
PASS

Air Blank Contamination Test
PASS

Range Exceeded Test
PASS


Duplicate Breath Sampling Results

Value	Test 1	Test 2	Control Test
.02	0.019	0.018	0.018
.04	0.038	0.038	0.039
.10	0.101	0.099	0.100
.20	0.199	0.196	0.195

PASS

CERTIFYING TECHNICIAN

I swear under penalty of perjury that in regards to the above listed instrument, I have complied with the Intoxilyzer Bench Checking Procedure approved by the State of Oklahoma.

SIGNATURE 	DATE 5/2/2022
NAME PAPE, CHRISTOPHER	PERMIT # 0000023020



**STATE OF OKLAHOMA
BOARD OF TESTS FOR ALCOHOL AND DRUG INFLUENCE**

Dr. Jarrad Wagner, Ph.D., F-ABFT
Chairman

Post Office Box 36307
Oklahoma City, Oklahoma 73136-2307
Phone: (405) 425-2460 Fax: (405) 425-2490
www.bot.ok.gov


Joshua Smith
Director

Certificate of Analysis – Dry Gas

ALPHA INSTRUMENT 2
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-004923
03/25/2022 15:00

Test	g/210L	Time
Air Blank	0.000	15:00
Cal Check	0.081	15:01
Air Blank	0.000	15:01
Cal Check	0.081	15:01
Air Blank	0.000	15:02
Cal Check	0.081	15:02
Air Blank	0.000	15:03
Cal Check	0.082	15:03
Air Blank	0.000	15:04
Cal Check	0.082	15:04
Air Blank	0.000	15:04
Cal Check	0.081	15:05
Air Blank	0.000	15:05
Cal Check	0.081	15:06
Air Blank	0.000	15:06
Cal Check	0.082	15:06
Air Blank	0.000	15:07
Cal Check	0.081	15:07
Air Blank	0.000	15:08
Cal Check	0.081	15:08
Air Blank	0.000	15:09
Cal Check Stats		
Average	0.0813	
Std Dev	0.0004	
Rel Std Dev(%)	0.5941	

**Known Breath-Alcohol Equivalent (g/210L) Ratio of Nitrogen and Ethanol Gas (208 ppm/105 L)
0.08 g/210L = 208 PPM ÷ 2605***

Manufacturer: ILMO	Manufacturer Cert. No.: 14549	Cylinder Size: 105L
Lot No.: 35521080A3	Cylinder No.: 011	Expiration Date: 03/05/2024
The above 105 L cylinder has been analyzed via Infrared Spectroscopy utilizing the Intoxilyzer 8000 and was performed in accordance with the procedures prescribed by the State Director of Tests pursuant to 40 OAC 30-1-3. The above NIST traceable reference standard meets the requirements for use as a calibrating unit in accordance with 47 O.S. 759, 40 OAC 25-1-3, and appears on the current conforming products list of calibrating units for breath alcohol testers published by the U.S. Department of Transportation in the Federal Register [77 FR 64588 et seq]. * See Dubowski, K.M. and Essary N., <i>Journal of Anal. Toxicol.</i> (1996) 20(6): pp. 484-491. [g/210L = PPM ÷ 2605].		
Signature 	Date: 03/25/2022	
Name Christopher Pape	Permit No.: 23020	



7 Eastgate Dr. • P.O. Box 790 • Jacksonville, IL 62651-0790
217-245-2183 • Fax: 217-243-7634 • www.ilmoproducts.com

Certificate of Analysis

Certificate ID: 14549
Part #: BAC105L080T
Cylinder Size: 105L
Lot Number: 35521080A3
Expiration: 3/5/2024

RCV'D
2/16/2022
CP

0.080 BAC (For the calibration of instruments used to determine breath alcohol concentration)

Contents: 105 Liters @ 1000 psig 70°F (21°C)

Component:	Reported Concentration:	Analytical Accuracy (U, k=2):	Analytical Method:
Ethanol	208 ppm	+/-0.002 BAC(G/210L)	NDIR
Nitrogen	Balance	[5.2 ppm]	

Distributed by:

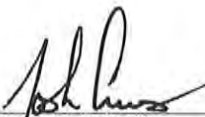
CMI Inc.
316 East Ninth Street
Owensboro, KY 42303
Phone 866-835-0690
www.alcoholtest.com

*Traceable to:

Certified Reference Material - 262.4 µmol/mol

Ethanol in Nitrogen - Serial No. GN0015026 Lot No. 050319E11

Store in dry area, away from sources of heat, ignition and direct sunlight. Do not allow storage area to exceed 52 °C (125 °F).


Specialty Gas Lab Tech

02-01-2022
Issuance Date



The calibration results within this certificate were obtained using equipment and standards capable of producing analytical results traceable to NIST, and apply only to the items contained on this certificate. ILMO Products Company makes no warranty or representation as to the suitability of the use of any information provided for any particular purpose. The information use is at the sole discretion and risk of the user. Liability shall be limited to established replacement cost of this material or service.

ISO/IEC 17025:2017 Accredited Laboratory

OFFICER'S AFFIDAVIT AND NOTICE OF REVOCATION/DISQUALIFICATION

7/17

Section 1

ARREST DATE 05/02/2022		ARREST TIME 1010		COUNTY # 55		CITY #		CITATION #	
ARREST LOCATION BOT						CITY		COUNTY OKLAHOMA	
SUBJECT NAME BREATHTEST, ALPHA				DATE OF BIRTH 01/01/1980		SEX M	HEIGHT 510	WEIGHT 150	
ADDRESS 3600 MARTIN LUTHER KING A				CITY OKLAHOMA CITY		STATE OK		ZIP CODE 73111	
DRIVER LICENSE # B083462869			EXPIRATION DATE 10/31/2022		STATE OK	CLASS D	RESTRICTIONS	ENDORSEMENTS	
VEHICLE MAKE MOCK			MODEL MOCK		TAG # MOCK		STATE OK	CMV N	PLACARD N

On the above date, time, and location, the above named person was arrested, and I had reasonable grounds to believe the person was driving or in actual physical control of a motor vehicle upon a public road, highway, street, turnpike, public parking lot, or other public place, or upon a private road, street, alley or lane which provides access to one or more single or multi-family dwelling within this state while under the influence of alcohol and/or other intoxicating substances as prohibited by law.

(Describe driving behavior or circumstances):

MOCK SUBJECT TEST


Describe person's condition (odor, actions, etc.):

POST REPAIR MAINTENANCE, INSTALLED NEW CYLINDER

THE PERSON WAS INFORMED OF THE IMPLIED CONSENT TEST REQUEST

BREATH-ALCOHOL ANALYSIS RECORD, REPORT OF THE PERSON NAMED IN SECTION 1 ABOVE, and LOG OF TEST AND MAINTENANCE RECORD.

Section 2

EQUIPMENT I-8000		SERIAL NUMBER 80-003394		SPECIALIST BOARD OF TESTS		
STANDARD Dry		TARGET VALUE 0.080		PERMIT NUMBER 23020		
MANUFACTURER ILMO		LOT # 35521080A3		DATE CYLINDER INSTALLED 05/02/2022	DATE CYLINDER EXPIRES 03/05/2024	
OBSERVATION START 1015	TEST DATE 05/02/2022	A 15-minute (or greater) deprivation/observation period was observed prior to and continuing to the time of test administration.				
RESULT TABLE		 SIGNATURE OF OPERATOR I administered said Breath-Alcohol Test in compliance with the applicable operating procedure of the Board of Tests.				
Test	g/210L					Time
Air Blank	0.00					10:48
Subject Test	0.00					10:48
Breath Volume	1.257 LITERS					
Air Blank	0.00					10:49
Wait						10:50
Air Blank	0.00					10:51
Subject Test	0.00					10:51
Breath Volume	1.785 LITERS					
Air Blank	0.00	10:51				
Cal Check	0.07	10:52				
Air Blank	0.00	10:52				



Test Result
BrAC g/210L
0.00

EFFECTIVE THIRTY (30) DAYS FROM DATE SERVED. Notice of Revocation is given that your driver's license is revoked or denied for a period of one hundred eighty (180) days or more, if you refused or failed the state's test(s). In addition, your commercial driving privilege in this state will be disqualified if you refused or failed the state's test(s) (operating a CMV or CDL holder) OR a test result of .04 or more while operating a commercial motor vehicle. You may appeal any Departmental action against your driver license as a result of this arrest by filing a petition in the District Court of the County in which you were arrested. Your petition must be filed within thirty (30) days of the date listed below. You may be eligible to participate in the Impaired Driver Accountability Program (IDAP) administered by the Department of Public Safety (DPS). Completion of IDAP may prevent a revocation as a result of this arrest from appearing on your driving record. Participation in IDAP may reduce the amount of fees you will be required to pay to the State. You must request IDAP within thirty (30) days of the date listed below. Mail the request for IDAP to the Department of Public Safety at P.O. Box 11415, Oklahoma City, OK 73136.

Section 3

SERVING OF NOTICE: I personally hand delivered a copy of the above containing the Notice of Revocation and the Notice of Department Impaired Driving Accountability Program to the person named in Section 1 above.

Date served: _____ Signature of officer: _____ Badge #: _____

In accordance with Title 12 O.S. Section 426, "I state under penalty of perjury under the laws of Oklahoma that the foregoing is true and correct."

Section 4

Date: _____ Place (location when signed): _____ Signature of arresting officer: _____

NAME PAPE, CHRISTOPHER		AGENCY BOARD OF TESTS	
BADGE # 0000000484	AGENCY ADDRESS INTER-AGENCY	ZIP CODE 00000	PHONE 4054252460

OTHER WITNESSES:

Section 5

1	NAME	TITLE	ADDRESS	PHONE
2	NAME	TITLE	ADDRESS	PHONE

OFFICER'S AFFIDAVIT AND NOTICE OF REVOCATION/DISQUALIFICATION

8/17

Section 1

ARREST DATE 05/02/2022		ARREST TIME 1010		COUNTY # 55	CITY #	CITATION #		
ARREST LOCATION BOT				CITY		COUNTY OKLAHOMA		
SUBJECT NAME BREATHTEST, ALPHA				DATE OF BIRTH 01/01/1980	SEX M	HEIGHT 510	WEIGHT 150	
ADDRESS 3600 MARTIN LUTHER KING A			CITY OKLAHOMA CITY		STATE OK	ZIP CODE 73111		
DRIVER LICENSE # B083462869			EXPIRATION DATE 10/31/2022	STATE OK	CLASS D	RESTRICTIONS	ENDORSEMENTS	
VEHICLE MAKE MOCK		MODEL MOCK		TAG # MOCK		STATE OK	CMV N	PLACARD N

On the above date, time, and location, the above named person was arrested, and I had reasonable grounds to believe the person was driving or in actual physical control of a motor vehicle upon a public road, highway, street, turnpike, public parking lot, or other public place, or upon a private road, street, alley or lane which provides access to one or more single or multi-family dwelling within this state while under the influence of alcohol and/or other intoxicating substances as prohibited by law.

(Describe driving behavior or circumstances):

MOCK SUBJECT TEST


Describe person's condition (odor, actions, etc.):

POST REPAIR MAINTENANCE, INSTALLED NEW CYLINDER

THE PERSON WAS INFORMED OF THE IMPLIED CONSENT TEST REQUEST

BREATH-ALCOHOL ANALYSIS RECORD, REPORT OF THE PERSON NAMED IN SECTION 1 ABOVE, and LOG OF TEST AND MAINTENANCE RECORD.

Section 2

EQUIPMENT I-8000		SERIAL NUMBER 80-003394	SPECIALIST BOARD OF TESTS	
STANDARD Dry		TARGET VALUE 0.080	PERMIT NUMBER 23020	
MANUFACTURER ILMO		LOT # 35521080A3	DATE CYLINDER INSTALLED 05/02/2022	DATE CYLINDER EXPIRES 03/05/2024
OBSERVATION START 1015	TEST DATE 05/02/2022	<p>A 15-minute (or greater) deprivation/observation period was observed prior to and continuing to the time of test administration.</p> <p>I administered said Breath-Alcohol Test in compliance with the applicable operating procedure of the Board of Tests.</p> <p style="text-align: center;"><i>Christopher Pape</i> SIGNATURE OF OPERATOR</p>		
RESULT TABLE				
Test	g/210L	Time	 <div style="border: 1px solid black; padding: 5px; text-align: center;"> Test Result BrAC g/210L 0.00 </div>	
Air Blank	0.00	10:48		
Subject Test	0.00	10:48		
Breath Volume	1.257 LITERS			
Air Blank	0.00	10:49		
Wait		10:50		
Air Blank	0.00	10:51		
Subject Test	0.00	10:51		
Breath Volume	1.785 LITERS			
Air Blank	0.00	10:51		
Cal Check	0.07	10:52		
Air Blank	0.00	10:52		
NAME PAPE, CHRISTOPHER		BADGE # 0000000484		
AGENCY BOARD OF TESTS		PERMIT # 0000023020		

EFFECTIVE THIRTY (30) DAYS FROM DATE SERVED. Notice of Revocation is given that your driver's license is revoked or denied for a period of one hundred eighty (180) days or more, if you refused or failed the state's test(s). In addition, your commercial driving privilege in this state will be disqualified if you refused or failed the state's test(s) (operating a CMV or CDL holder) OR a test result of .04 or more while operating a commercial motor vehicle. You may appeal any Departmental action against your driver license as a result of this arrest by filing a petition in the District Court of the County in which you were arrested. Your petition must be filed within thirty (30) days of the date listed below. You may be eligible to participate in the Impaired Driver Accountability Program (IDAP) administered by the Department of Public Safety (DPS). Completion of IDAP may prevent a revocation as a result of this arrest from appearing on your driving record. Participation in IDAP may reduce the amount of fees you will be required to pay to the State. You must request IDAP within thirty (30) days of the date listed below. Mail the request for IDAP to the Department of Public Safety at P.O. Box 11415, Oklahoma City, OK 73136.

Section 3

SERVING OF NOTICE: I personally hand delivered a copy of the above containing the Notice of Revocation and the Notice of Department Impaired Driving Accountability Program to the person named in Section 1 above.

Date served: _____ Signature of officer: _____ Badge #: _____

In accordance with Title 12 O.S. Section 426, "I state under **penalty of perjury** under the laws of Oklahoma that the foregoing is true and correct."

Section 4

Date: _____ Place (location when signed): _____ Signature of arresting officer: _____

NAME PAPE, CHRISTOPHER		AGENCY BOARD OF TESTS	
BADGE 0000000484	AGENCY ADDRESS INTER-AGENCY	ZIP CODE 00000	PHONE 4054252460



9/17

STATE OF OKLAHOMA BOARD OF TESTS FOR ALCOHOL AND DRUG INFLUENCE

Dr. Jarrad Wagner, Ph.D., F-ABFT
Chairman

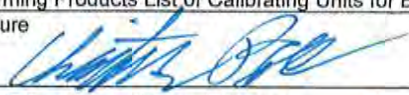
Post Office Box 36307
Oklahoma City, Oklahoma 73136-2307
Phone: (405) 425-2460 Fax: (405) 425-2490
www.bot.ok.gov

Joshua Smith
Director

Certificate of Analysis – Reference Solution

ALPHA INSTRUMENT 2
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-004923
03/08/2022 11:33

Test	g/210L	Time
Air Blank	0.000	11:34
Cal Check	0.022	11:34
Air Blank	0.000	11:35
Cal Check	0.021	11:36
Air Blank	0.000	11:36
Cal Check	0.021	11:37
Air Blank	0.000	11:37
Cal Check	0.021	11:38
Air Blank	0.000	11:38
Cal Check	0.021	11:39
Air Blank	0.000	11:40
Cal Check	0.021	11:40
Air Blank	0.000	11:41
Cal Check	0.020	11:42
Air Blank	0.000	11:42
Cal Check	0.021	11:43
Air Blank	0.000	11:43
Cal Check	0.021	11:44
Air Blank	0.000	11:45
Cal Check	0.020	11:45
Air Blank	0.000	11:46
Cal Check Stats		
Average	0.0209	
Std Dev	0.0005	
Rel Std Dev(%)	2.7158	

NIST Traceable Breath-Alcohol Equivalent (g/210L +/- 3%) Ratio of ethanol and water free of test interfering substances.		
Manufacturer: Guth	Manufactured Date: 04/07/2021	Concentration: 0.02
Lot No.: 21120	Expiration Date: 04/07/2023	Equipment Model: Guth 2100
The above solution has been analyzed via Infrared Spectroscopy utilizing the Intoxilyzer 8000 and was performed in accordance with the procedures prescribed by the State Director of Tests pursuant to 40 OAC 30-1-3. The above NIST traceable reference solution meets the requirements for use pursuant to 40 OAC 25-1-2.1. Approved breath-alcohol measurement equipment and reference solutions. Equipment used appears on the current or supplemented Conforming Products List of Calibrating Units for Breath Alcohol Testers (72 FR 34747), published by the National Highway Traffic Safety Administration.		
Signature 	Date: 03/08/2022	
Name Christopher Pape	Permit No.: 23020	



10/17

STATE OF OKLAHOMA BOARD OF TESTS FOR ALCOHOL AND DRUG INFLUENCE

Dr. Jarrad Wagner, Ph.D., F-ABFT
Chairman

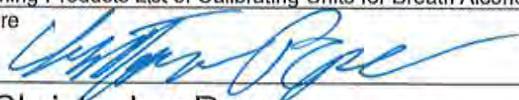
Post Office Box 36307
Oklahoma City, Oklahoma 73136-2307
Phone: (405) 425-2460 Fax: (405) 425-2490
www.bot.ok.gov

Joshua Smith
Director

Certificate of Analysis – Reference Solution

ALPHA INSTRUMENT 2
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-004923
03/08/2022 11:53

Test	g/210L	Time
Air Blank	0.000	11:53
Cal Check	0.041	11:54
Air Blank	0.000	11:54
Cal Check	0.041	11:55
Air Blank	0.000	11:56
Cal Check	0.042	11:56
Air Blank	0.000	11:57
Cal Check	0.042	11:58
Air Blank	0.000	11:58
Cal Check	0.042	11:59
Air Blank	0.000	11:59
Cal Check	0.041	12:00
Air Blank	0.000	12:01
Cal Check	0.041	12:01
Air Blank	0.000	12:02
Cal Check	0.041	12:02
Air Blank	0.000	12:03
Cal Check	0.041	12:04
Air Blank	0.000	12:04
Cal Check	0.042	12:05
Air Blank	0.000	12:05
Cal Check Stats		
Average	0.0414	
Std Dev	0.0005	
Rel Std Dev(%)	1.2473	

NIST Traceable Breath-Alcohol Equivalent (g/210L +/- 3%) Ratio of ethanol and water free of test interfering substances.		
Manufacturer: Guth	Manufactured Date: 03/01/2021	Concentration: 0.04
Lot No.: 21070	Expiration Date: 03/01/2023	Equipment Model: Guth 2100
The above solution has been analyzed via Infrared Spectroscopy utilizing the Intoxilyzer 8000 and was performed in accordance with the procedures prescribed by the State Director of Tests pursuant to 40 OAC 30-1-3. The above NIST traceable reference solution meets the requirements for use pursuant to 40 OAC 25-1-2.1. Approved breath-alcohol measurement equipment and reference solutions. Equipment used appears on the current or supplemented Conforming Products List of Calibrating Units for Breath Alcohol Testers (72 FR 34747), published by the National Highway Traffic Safety Administration.		
Signature 	Date: 03/08/2022	
Name Christopher Pape	Permit No.: 23020	



STATE OF OKLAHOMA BOARD OF TESTS FOR ALCOHOL AND DRUG INFLUENCE

Dr. Jarrad Wagner, Ph.D., F-ABFT
Chairman

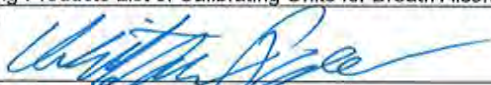
Post Office Box 36307
Oklahoma City, Oklahoma 73136-2307
Phone: (405) 425-2460 Fax: (405) 425-2490
www.bot.ok.gov

Joshua Smith
Director

Certificate of Analysis – Reference Solution

ALPHA INSTRUMENT 2
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-004923
03/08/2022 12:11

Test	g/210L	Time
Air Blank	0.000	12:11
Cal Check	0.101	12:12
Air Blank	0.000	12:13
Cal Check	0.101	12:13
Air Blank	0.000	12:14
Cal Check	0.101	12:15
Air Blank	0.000	12:15
Cal Check	0.102	12:16
Air Blank	0.000	12:16
Cal Check	0.101	12:17
Air Blank	0.000	12:18
Cal Check	0.101	12:18
Air Blank	0.000	12:19
Cal Check	0.101	12:20
Air Blank	0.000	12:20
Cal Check	0.101	12:21
Air Blank	0.000	12:21
Cal Check	0.101	12:22
Air Blank	0.000	12:23
Cal Check	0.101	12:23
Air Blank	0.000	12:24
Cal Check Stats		
Average	0.1011	
Std Dev	0.0003	
Rel Std Dev(%)	0.3127	

NIST Traceable Breath-Alcohol Equivalent (g/210L +/- 3%) Ratio of ethanol and water free of test interfering substances.		
Manufacturer: Guth	Manufactured Date: 06/08/2021	Concentration: 0.10
Lot No.: 21190	Expiration Date: 06/08/2023	Equipment Model: Guth 2100
The above solution has been analyzed via Infrared Spectroscopy utilizing the Intoxilyzer 8000 and was performed in accordance with the procedures prescribed by the State Director of Tests pursuant to 40 OAC 30-1-3. The above NIST traceable reference solution meets the requirements for use pursuant to 40 OAC 25-1-2.1. Approved breath-alcohol measurement equipment and reference solutions. Equipment used appears on the current or supplemented Conforming Products List of Calibrating Units for Breath Alcohol Testers (72 FR 34747), published by the National Highway Traffic Safety Administration.		
Signature 	Date: 03/08/2022	
Name Christopher Pape	Permit No.: 23020	



12/17

STATE OF OKLAHOMA BOARD OF TESTS FOR ALCOHOL AND DRUG INFLUENCE

Dr. Jarrad Wagner, Ph.D., F-ABFT
Chairman

Post Office Box 36307
Oklahoma City, Oklahoma 73136-2307
Phone: (405) 425-2460 Fax: (405) 425-2490
www.bot.ok.gov

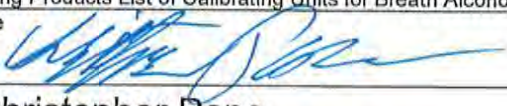
Joshua Smith
Director

Certificate of Analysis – Reference Solution

ALPHA INSTRUMENT 2
Intoxilyzer - Alcohol Analyzer
Model 8000 SN 80-004923
03/08/2022 12:28

Test	g/210L	Time
Air Blank	0.000	12:29
Cal Check	0.201	12:29
Air Blank	0.000	12:30
Cal Check	0.200	12:31
Air Blank	0.000	12:31
Cal Check	0.201	12:32
Air Blank	0.000	12:33
Cal Check	0.201	12:33
Air Blank	0.000	12:34
Cal Check	0.201	12:35
Air Blank	0.000	12:35
Cal Check	0.201	12:36
Air Blank	0.000	12:36
Cal Check	0.201	12:37
Air Blank	0.000	12:38
Cal Check	0.200	12:38
Air Blank	0.000	12:39
Cal Check	0.201	12:40
Air Blank	0.000	12:40
Cal Check	0.200	12:41
Air Blank	0.000	12:41
Cal Check Stats		
Average	0.2007	
Std Dev	0.0004	
Rel Std Dev(%)	0.2406	

NIST Traceable Breath-Alcohol Equivalent (g/210L +/- 3%) Ratio of ethanol and water free of test interfering substances.

Manufacturer: Guth	Manufactured Date: 12/03/2020	Concentration: 0.20
Lot No.: 20510	Expiration Date: 12/03/2022	Equipment Model: Guth 2100
The above solution has been analyzed via Infrared Spectroscopy utilizing the Intoxilyzer 8000 and was performed in accordance with the procedures prescribed by the State Director of Tests pursuant to 40 OAC 30-1-3. The above NIST traceable reference solution meets the requirements for use pursuant to 40 OAC 25-1-2.1. Approved breath-alcohol measurement equipment and reference solutions. Equipment used appears on the current or supplemented Conforming Products List of Calibrating Units for Breath Alcohol Testers (72 FR 34747), published by the National Highway Traffic Safety Administration.		
Signature 	Date: 03/08/2022	
Name Christopher Pape	Permit No.: 23020	

**GUTH LABORATORIES, INC.**

590 NORTH 67th STREET • HARRISBURG, PA 17111-4511 • TELEPHONE: 717-564-5470

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 21120 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on April 16, 2021, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.0237% (w/vol) ethyl alcohol. The expiration date for this lot number is April 7, 2023 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.020 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President
GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN08031603 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.

RCV'D
8/26/21
CP

**GUTH LABORATORIES, INC.**

590 NORTH 67th STREET • HARRISBURG, PA 17111-4811 • TELEPHONE: 717-564-5470

CERTIFICATE OF ANALYSISACV'D
8/26/21
CP

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number **21070** of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on **March 2, 2021**, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain **0.0488%** (w/vol) ethyl alcohol. The expiration date for this lot number is **March 1, 2023** at 11:59 PM.

When used in a calibrated Simulator, operating at $34^{\circ}\text{C} \pm .2^{\circ}\text{C}$, this solution will give a breath alcohol analysis instrument reading of **0.040 g/210L \pm 3%**.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President
GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN08211802 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.

**GUTH LABORATORIES, INC.**

590 NORTH 67th STREET • HARRISBURG, PA 17111-4511 • TELEPHONE: 717-564-5470

CERTIFICATE OF ANALYSISRCV'D
8/26/21
CP**Certified Alcohol Reference Solution for Simulator**

Random Samples of Lot Number 21190 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on June 9, 2021, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1206% (w/vol) ethyl alcohol. The expiration date for this lot number is June 8, 2023 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President
GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN03052002 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.

**GUTH LABORATORIES, INC.**

590 NORTH 67th STREET • HARRISBURG, PA 17111-4511 • TELEPHONE: 717-564-5470

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number **20510** of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on December 7, 2020, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.2419% (w/vol) ethyl alcohol. The expiration date for this lot number is December 3, 2022 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.200 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President
GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN06231703 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.

RCV'D
8/26/21
CP

Location Returned From Ft. Gibson PD	Unit Serial # 80-003394	Date: 17/17 4/26/22
---	----------------------------	------------------------

Inventory Process

Old Cylinder #: 07220080A2-004

Unit checked in to BOT
 Unlink Gas Canister from Unit
 Old cylinder disposed in inventory
 New Unit Checked out
 Remove old cylinder from unit (After Lab Evaluation)
 Old cylinder archived

Check in Procedure

Connect unit to ethernet connection 172.019.004. 130
 Download any stored records 14
 Change unit location to Serial number
 Update Unit Date and time
 Previous time: 4/28/22 0916
 New Time: 4/28/22 0920
 Execute a new Forms load on the unit

Notes from the Field

Routine Swap
 Issues in the Field
Purge Fail after lightning strike

Lab Evaluation

Breath Hose Hold Vacuum?: Test Chamber Hold Vacuum?:

DVM

3µm 12770 Chamber Temp 47.00 Snap Test ok?
 9µm 13739 Breath Hose Temp 45 RFI Detected?

Peripheral Tests

Speaker (F5) Power LED Green (F6) Red (F7) Fan (F9)
 Gas Regulator (F11) Pump (F10) Receipt paper

Atmospheric Sensor

Pressure 977 Correction Factor 1.03 Flow Sensor Resistance 220 Tank Sensor Value 280
 Is regulator close to Tank Sensor Value? Yes / No

Sent to CMI (If needed)	Reason Sent
Date Sent _____	
Invoice Sent _____	
Date Returned _____	

Repairs needed:

Repairs Made:

Replaced check valve

Ready for Bench Check?

<u>Inventory Process</u> <input type="checkbox"/> Linked Cylinder to Unit <input type="checkbox"/> Changed Unit Status <input type="checkbox"/> Bench Check Report Completed	<input type="checkbox"/> Bench Check Technician Report <input type="checkbox"/> Mock Subject Test Completed <input type="checkbox"/> Certificate of Calibration and Operation <input type="checkbox"/> Service Overview Completed	<u>To complete if unit sent to CMI</u> <input type="checkbox"/> CMI Workorder <input type="checkbox"/> CMI Certificate of Calibration _____ <input type="checkbox"/> CMI Service Evaluation Form
---	--	---