

Service Overview

Board of Tests for Alcohol and Drug Influence
*Enhancing public safety by ensuring the accuracy and scientific reliability
of blood and breath testing*



3600 N. Martin Luther King Bldg #9
Oklahoma City, OK 73111
405-425-2460

BENCH CHECK DATE: 09-25-2017

SERIAL: 80-003403

DOCUMENTATION WITHIN SERVICE PACKET:

- BOT Certificate of Calibration and Operation
- Bench Check
- Bench Check Technician's Report
- BOT Dry Gas Certificate of Analysis
- Mock Subject Test
- BOT Certificates of Analysis on Certified Solutions (4)

Date:

Re: Request for Log of Test on Intoxilyzer 8000

Name:

Dear Sir:

The Log of Test information from a breath test conducted on a Intoxilyzer 8000 is printed in Section 2 of the Affidavit and is no longer retained as a separate form as with the Intoxilyzer 5000. The 8000 uses a nitrogen/ethanol dry gas for the external control test. The dry gas cylinder has the capacity to perform approximately 325 tests and does not require maintenance until its expiration date or can no longer provide the required pressure to perform a control test. The cylinder has an expiration date of two years from date filled at the manufacturer and is programmed in the 8000 upon installation, both dates are printed in Section 2 of the Affidavit. The Intoxilyzer is programmed to disable itself when the date of expiration is reached or the pressure monitor indicates 25 lbs psi.

Due to the reliability and stability of dry gas, maintenance on the 8000 is not required every 30 days or 25 tests, therefore maintenance is no longer performed by officers in the field. The Board of Tests for Alcohol and Drug Influence certifies each Intoxilyzer 8000 for field use by performing a bench check. The bench check is designed to check the operation and procedures programmed in the 8000 to ensure it is functioning properly for use in Oklahoma. A bench check is performed every time a new dry gas cylinder is installed or the instrument has returned from the manufacturer for repair. Once the instrument passes the bench check, all that is required to set it up for field use is to plug it in and turn it on and does not require a permit to set up. The Board of Tests has the ability to monitor each instrument once it is set up in the field.

Each Intoxilyzer has a database of valid permit numbers that it checks prior to allowing a breath test to be performed. The database is updated on a regular basis, if a permit number is invalid that officer will not be able to conduct a breath test on the 8000. If the 8000 prints an affidavit the breath test operator listed in Section 2 had a valid permit at the time of the test and the breath test is valid.

If you have any further questions please feel free to call me.

Sincerely,



J. Robert Blakeburn (OBA11059)
State Director of Tests for Alcohol
and Drug Influence

**STATE OF OKLAHOMA - DEPARTMENT OF PUBLIC SAFETY
CMI INTOXILYZER BENCH CHECK PROCEDURE REPORT**

| | | |
|--------------------------------|-------------------------------|---------------------------------------|
| BENCH CHECK DATE 09/25/2017 | TIME PROCEDURE BEGAN 10:25 | INSTRUMENT LOCATION 80-003403 |
| INSTRUMENT TYPE Intoxilyzer | INSTRUMENT MODEL I-8000 | INSTRUMENT SERIAL NUMBER 80-003403 |

TEST RESULTS

Diagnostics

PASS

Mouth Alcohol Test

PASS

RFI Test

PASS

Abort Test

PASS

Improper Sample Test

PASS

Interferent Detect Test

PASS

No Sample Given Test

PASS

Insufficient Sample Test

PASS

Air Blank Contamination Test

PASS

Range Exceeded Test

PASS


Duplicate Breath Sampling Results

| Value | Test 1 | Test 2 | Control Test |
|-------|--------|--------|--------------|
| .02 | 0.018 | 0.017 | 0.019 |
| .04 | 0.038 | 0.038 | 0.039 |
| .10 | 0.097 | 0.097 | 0.098 |
| .20 | 0.196 | 0.197 | 0.200 |

PASS

CERTIFYING TECHNICIAN

I swear under penalty of perjury that in regards to the above listed instrument, I have complied with the Intoxilyzer Bench Checking Procedure approved by the State of Oklahoma.

| | |
|--|------------------------|
| SIGNATURE  | DATE 09/25/17 |
| NAME SMITH, JOSHUA | PERMIT # 0000016837 |



3600 N. Martin Luther King Bldg #9
 Oklahoma City, OK 73111
 405-425-2460

BENCH CHECK TECHNICIAN'S REPORT

Date: 9/25/17 Start Time: 0950 End Time: 1230

INSTRUMENT

| | |
|-------------------|-----------|
| Intoxilyzer Model | Serial # |
| 8000 | 80-003403 |

GAS CANISTER

| | |
|-----------------|----------|
| LOT # | EXP Date |
| 19B17080A2 #844 | 9-5-2019 |

REFERENCE

| Simulator Model | | | |
|--------------------------|------------|------------|------------|
| Guth 2100 | Guth 2100 | Guth 2100 | Guth 2100 |
| Serial # | | | |
| DR3591 | DR3753 | DR3594 | DR3754 |
| Concentration | | | |
| .02 | .04 | .10 | .20 |
| LOT # | | | |
| 16400 | 16320 | 17130 | 17090 |
| Manufactured Date | | | |
| 12-12-2016 | 10-21-2016 | 04-03-2017 | 02-24-2017 |
| Expiration Date | | | |
| 12-12-2018 | 10-21-2018 | 04-03-2019 | 02-24-2019 |
| Solution Commission Date | | | |
| 9-18-17 | 9-18-17 | 9-18-17 | 9-18-17 |

REASON FOR BENCH CHECK

- POST REPAIR
 CYLINDER REPLACEMENT
 TROUBLESHOOTING
 ROUTINE MAINTENANCE

COMMENTS

| |
|--|
| |
| |
| |
| |

Technician Signature / Permit #

[Handwritten Signature]
 Josh Smith

Printed Name

| Certificate of Analysis – Dry Gas | |
|-----------------------------------|--------------------------------|
| 9-5-2019 | 0.080 |
| Date of Analysis | Labelled target value (g/210L) |
| 044 | 0.0802 |
| Cylinder # | Average test result |
| Josh Smith <i>[Signature]</i> | |
| BOT Technician Name and Signature | |



ALPHA INSTRUMENT
 Intoxilyzer - Alcohol Analyzer
 Model 8000 SN 80-002591
 09/18/2017 13:54

ISO/IEC 17025:2005 Accredited Laboratory
BRT Form 500

In accordance with BOT Policy and Procedure Statement BRT-2.1.0, *Maintenance*, the above referenced dry gas cylinder is suitable for use as an external reference with the Intoxilyzer 8000.

| Test | g/210L | Time |
|-----------------|--------|-------|
| Air Blank | 0.000 | 13:54 |
| Cal Check | 0.080 | 13:54 |
| Air Blank | 0.000 | 13:55 |
| Cal Check | 0.080 | 13:55 |
| Air Blank | 0.000 | 13:56 |
| Cal Check | 0.080 | 13:56 |
| Air Blank | 0.000 | 13:57 |
| Cal Check | 0.080 | 13:57 |
| Air Blank | 0.000 | 13:58 |
| Cal Check | 0.080 | 13:58 |
| Air Blank | 0.000 | 13:58 |
| Cal Check | 0.081 | 13:59 |
| Air Blank | 0.000 | 13:59 |
| Cal Check | 0.080 | 14:00 |
| Air Blank | 0.000 | 14:00 |
| Cal Check | 0.081 | 14:00 |
| Air Blank | 0.000 | 14:01 |
| Cal Check | 0.080 | 14:01 |
| Air Blank | 0.000 | 14:02 |
| Cal Check | 0.080 | 14:02 |
| Air Blank | 0.000 | 14:03 |
| Cal Check Stats | | |
| Average | 0.0802 | |
| Std Dev | 0.0004 | |
| Rel Std Dev(%) | 0.5257 | |

Certificate of Analysis

Certificate ID: 10399 ✓
Part #: BAC105L080T
Cylinder Size: 105L
Lot Number: 19817080A2 ✓
Expiration: 9/5/2019 ✓

Rec'd 09/18/17
[Signature]

0.080 BAC (For the calibration of instruments used to determine breath alcohol concentration)

Contents: 105 Liters @ 1000 psig 70°F (21°C)

| Component: | Concentration: | Accuracy: | Method: |
|------------|----------------|--------------------------|---------|
| Ethanol | 208 ppm | +/- 0.002 or 2% | NDIR |
| Nitrogen | Balance | BAC whichever is greater | |

[Signature]

 Operator's Signature

*NIST Standard Reference Material
 Cylinder No. CC 274507 / Job No. 09160309
 Certified 362.2 µmol/mol Ethanol in Nitrogen
 for ILMO Products Co., Jacksonville, IL

Store in dry area, away from sources of heat, ignition and direct sunlight. Do not allow storage area to exceed 52 °C (125 °F).

[Signature]
 Specialty Gas Lab Tech

08-16-17
 Date

Distributed by:

CMI Inc.
 316 East Ninth Street
 Owensboro, KY 42303
 Phone 866-835-0690
www.alcoholtest.com



OFFICER'S AFFIDAVIT AND NOTICE OF REVOCAION/DISQUALIFICATION

Section 1

| | | | | | | | |
|---|--|------------------------------|--|--------------------------------------|---------------------|---------------------------|--------------------------|
| ARREST DATE 09/25/2017 | | ARREST TIME 1030 | | COUNTY # 55 | CITY # 55 | CITATION # 2017 | |
| ARREST LOCATION MOCK | | | | CITY MOCK | | COUNTY OKLAHOMA | |
| SUBJECT NAME MOCK, SUBJECT TEST | | | | DATE OF BIRTH 08/09/1961 | | SEX M | RACE W |
| ADDRESS 3600 MARTIN LUTHER KING A | | CITY OKLAHOMA CITY | | | STATE OK | | ZIP CODE 73133 |
| DRIVER LICENSE # T083454871 | | | | EXPIRATION DATE 06/30/2019 | | STATE OK | CLASS A |
| VEHICLE MAKE MOCK | | MODEL MOCK | | TAG # MOCK | | STATE OK | CMV N |
| | | | | | | RESTRICTIONS | ENDORSEMENTS M |

On the above date, time, and location, the above named person was arrested, and I had reasonable grounds to believe the person was driving or in actual physical control of a motor vehicle upon a public road, highway, street, turnpike, public parking lot, or other public place, or upon a private road, street, alley or lane which provides access to one or more single or multi-family dwelling within this state while under the influence of alcohol and/or other intoxicating substances as prohibited by law.

(Describe driving behavior or circumstances):

MOCK SUBJECT TEST. ROUTINE MAINTENANCE. NEW GAS CANISTER.

Describe person's condition (odor, actions, etc.):

NONE

THE PERSON WAS INFORMED OF THE IMPLIED CONSENT TEST REQUEST

BREATH-ALCOHOL ANALYSIS RECORD, REPORT OF THE PERSON NAMED IN SECTION 1 ABOVE, and LOG OF TEST AND MAINTENANCE RECORD.

Section 2

| EQUIPMENT I-8000 | | SERIAL NUMBER 80-003403 | SPECIALIST BOARD OF TESTS | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
|--|--------------------------------|-----------------------------------|--|--|------|--------|------|-----------|------|-------|--------------|------|-------|---------------|--------------|--|-----------|------|-------|------|--|-------|-----------|------|-------|--------------|------|-------|---------------|--------------|--|-----------|------|-------|-----------|------|-------|-----------|------|-------|
| STANDARD Dry | | TARGET VALUE 0.080 | PERMIT NUMBER 16837 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| MANUFACTURER ILMO | | LOT # 19817080A2 | DATE CYLINDER INSTALLED 09/25/2017 | DATE CYLINDER EXPIRES 09/05/2019 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| OBSERVATION START 1120 | TEST DATE 09/25/2017 | MOUTHPIECE G | <p style="text-align: center;">A 15-minute (or greater) deprivation/observation period was observed prior to and continuing to the time of test administration.</p> <p style="text-align: center;">I administered said Breath-Alcohol Test in compliance with the applicable operating procedure of the Board of Tests.</p> <p style="text-align: center;">_____ SIGNATURE OF OPERATOR</p> | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| <table border="1" style="width: 100%; border-collapse: collapse;"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr> <td>Air Blank</td> <td>0.00</td> <td>11:55</td> </tr> <tr> <td>Subject Test</td> <td>0.00</td> <td>11:55</td> </tr> <tr> <td style="padding-left: 20px;">Breath Volume</td> <td>1.628 LITERS</td> <td></td> </tr> <tr> <td>Air Blank</td> <td>0.00</td> <td>11:56</td> </tr> <tr> <td>Wait</td> <td></td> <td>11:58</td> </tr> <tr> <td>Air Blank</td> <td>0.00</td> <td>11:58</td> </tr> <tr> <td>Subject Test</td> <td>0.00</td> <td>12:00</td> </tr> <tr> <td style="padding-left: 20px;">Breath Volume</td> <td>1.222 LITERS</td> <td></td> </tr> <tr> <td>Air Blank</td> <td>0.00</td> <td>12:00</td> </tr> <tr> <td>Cal Check</td> <td>0.08</td> <td>12:00</td> </tr> <tr> <td>Air Blank</td> <td>0.00</td> <td>12:01</td> </tr> </tbody> </table> | | | | | Test | g/210L | Time | Air Blank | 0.00 | 11:55 | Subject Test | 0.00 | 11:55 | Breath Volume | 1.628 LITERS | | Air Blank | 0.00 | 11:56 | Wait | | 11:58 | Air Blank | 0.00 | 11:58 | Subject Test | 0.00 | 12:00 | Breath Volume | 1.222 LITERS | | Air Blank | 0.00 | 12:00 | Cal Check | 0.08 | 12:00 | Air Blank | 0.00 | 12:01 |
| Test | g/210L | Time | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Air Blank | 0.00 | 11:55 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Subject Test | 0.00 | 11:55 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Breath Volume | 1.628 LITERS | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Air Blank | 0.00 | 11:56 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Wait | | 11:58 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Air Blank | 0.00 | 11:58 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Subject Test | 0.00 | 12:00 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Breath Volume | 1.222 LITERS | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Air Blank | 0.00 | 12:00 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Cal Check | 0.08 | 12:00 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Air Blank | 0.00 | 12:01 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| RESULT TABLE | | | NAME SMITH, JOSHUA | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| | | | BADGE # 0000000435 | PERMIT # 0000016837 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| | | | AGENCY BOARD OF TESTS | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |

EFFECTIVE THIRTY (30) DAYS FROM DATE SERVED. Notice of Revocation is given that your driver's license is revoked or denied for a period of one hundred eighty (180) days or more, if you refused or failed the state's test(s). In addition, your commercial driving privilege in this state will be disqualified if you refused or failed the state's test(s) (operating a CMV or CDL holder) OR a test result of .04 or more while operating a commercial motor vehicle. To appeal the driver's license revocation/disqualification, the Department of Public Safety must receive your written request for an Administrative Hearing **within fifteen (15) days from the date you received notice of revocation or disqualification**. Submit your **written** request in person at 3600 N. MLK Ave. or by mail to Legal Division, Department of Public Safety, P.O. Box 11415, OKC, OK, 73136

Section 3

SERVING OF NOTICE: I personally hand-delivered a copy of the above containing the Notice of Revocation to the person named in Section 1 above.

Date served: **09/25/2017** Signature of officer: _____ Badge #: _____

In accordance with Title 12 O.S. Section 426, "I state under **penalty of perjury** under the laws of Oklahoma that the foregoing is true and correct." Section 4

Date: **09/25/2017** Place (location when signed): _____ Signature of arresting officer: _____

| | | | |
|------------------------------|---------------------------------------|---------------------------------|------------------------------|
| NAME SMITH, JOSHUA | | AGENCY BOARD OF TESTS | |
| BADGE 0000000435 | AGENCY ADDRESS INTER-AGENCY | ZIP CODE 00000 | PHONE "4054252460" |

No Temporary License Issued:
Test Below Legal Limit

ARRESTEE'S COPY
BOT Form AFF02 07/2008

OFFICER'S AFFIDAVIT AND NOTICE OF REVOCATION/DISQUALIFICATION

Section 1

| | | | | | | | |
|---|--|------------------------------|--|--------------------------------------|---------------------|---------------------------|-------------------------|
| ARREST DATE 09/25/2017 | | ARREST TIME 1030 | | COUNTY # 55 | CITY # 55 | CITATION # 2017 | |
| ARREST LOCATION MOCK | | | | CITY MOCK | | COUNTY OKLAHOMA | |
| SUBJECT NAME MOCK, SUBJECT TEST | | | | DATE OF BIRTH 08/09/1961 | SEX M | RACE W | HEIGHT 510 |
| ADDRESS 3600 MARTIN LUTHER KING A | | CITY OKLAHOMA CITY | | | STATE OK | ZIP CODE 73133 | |
| DRIVER LICENSE # T083454871 | | | | EXPIRATION DATE 06/30/2019 | STATE OK | CLASS A | RESTRICTIONS |
| VEHICLE MAKE MOCK | | MODEL MOCK | | TAG # MOCK | | STATE OK | CMV N |
| | | | | | | PLACARD N | |

On the above date, time, and location, the above named person was arrested, and I had reasonable grounds to believe the person was driving or in actual physical control of a motor vehicle upon a public road, highway, street, turnpike, public parking lot, or other public place, or upon a private road, street, alley or lane which provides access to one or more single or multi-family dwelling within this state while under the influence of alcohol and/or other intoxicating substances as prohibited by law.

(Describe driving behavior or circumstances):

MOCK SUBJECT TEST. ROUTINE MAINTENANCE. NEW GAS CANISTER.

Describe person's condition (odor, actions, etc.):

NONE

THE PERSON WAS INFORMED OF THE IMPLIED CONSENT TEST REQUEST

BREATH-ALCOHOL ANALYSIS RECORD, REPORT OF THE PERSON NAMED IN SECTION 1 ABOVE, and LOG OF TEST AND MAINTENANCE RECORD.

Section 2

| EQUIPMENT I-8000 | | SERIAL NUMBER 80-003403 | SPECIALIST BOARD OF TESTS | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
|--|--------------------------------|-----------------------------------|--|--|------|--------|------|-------|--|--|-----------|------|-------|--------------|------|-------|---------------|--------------|--|-----------|------|-------|------|--|-------|-----------|------|-------|--------------|------|-------|---------------|--------------|--|-----------|------|-------|-----------|------|-------|-----------|------|-------|
| STANDARD Dry | | TARGET VALUE 0.080 | PERMIT NUMBER 16837 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| MANUFACTURER ILMO | | LOT # 19817080A2 | DATE CYLINDER INSTALLED 09/25/2017 | DATE CYLINDER EXPIRES 09/05/2019 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| OBSERVATION START 1120 | TEST DATE 09/25/2017 | MOUTHPIECE G | <p style="text-align: center;">A 15-minute (or greater) deprivation/observation period was observed prior to and continuing to the time of test administration.</p> <p style="text-align: center;">I administered said Breath-Alcohol Test in compliance with the applicable operating procedure of the Board of Tests.</p> <p style="text-align: center;">SIGNATURE OF OPERATOR _____</p> | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| <table border="1" style="width: 100%; border-collapse: collapse;"> <thead> <tr> <th>Test</th> <th>g/210L</th> <th>Time</th> </tr> </thead> <tbody> <tr><td colspan="3">-----</td></tr> <tr><td>Air Blank</td><td>0.00</td><td>11:55</td></tr> <tr><td>Subject Test</td><td>0.00</td><td>11:55</td></tr> <tr><td> Breath Volume</td><td>1.628 LITERS</td><td></td></tr> <tr><td>Air Blank</td><td>0.00</td><td>11:56</td></tr> <tr><td>Wait</td><td></td><td>11:58</td></tr> <tr><td>Air Blank</td><td>0.00</td><td>11:58</td></tr> <tr><td>Subject Test</td><td>0.00</td><td>12:00</td></tr> <tr><td> Breath Volume</td><td>1.222 LITERS</td><td></td></tr> <tr><td>Air Blank</td><td>0.00</td><td>12:00</td></tr> <tr><td>Cal Check</td><td>0.08</td><td>12:00</td></tr> <tr><td>Air Blank</td><td>0.00</td><td>12:01</td></tr> </tbody> </table> | | | | | Test | g/210L | Time | ----- | | | Air Blank | 0.00 | 11:55 | Subject Test | 0.00 | 11:55 | Breath Volume | 1.628 LITERS | | Air Blank | 0.00 | 11:56 | Wait | | 11:58 | Air Blank | 0.00 | 11:58 | Subject Test | 0.00 | 12:00 | Breath Volume | 1.222 LITERS | | Air Blank | 0.00 | 12:00 | Cal Check | 0.08 | 12:00 | Air Blank | 0.00 | 12:01 |
| Test | g/210L | Time | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| ----- | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Air Blank | 0.00 | 11:55 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Subject Test | 0.00 | 11:55 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Breath Volume | 1.628 LITERS | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Air Blank | 0.00 | 11:56 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Wait | | 11:58 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Air Blank | 0.00 | 11:58 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Subject Test | 0.00 | 12:00 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Breath Volume | 1.222 LITERS | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Air Blank | 0.00 | 12:00 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Cal Check | 0.08 | 12:00 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| Air Blank | 0.00 | 12:01 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| RESULT TABLE | | | NAME SMITH, JOSHUA | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| | | | BADGE # 0000000435 | PERMIT # 0000016837 | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |
| | | | AGENCY BOARD OF TESTS | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | | |

EFFECTIVE THIRTY (30) DAYS FROM DATE SERVED. Notice of Revocation is given that your driver's license is revoked or denied for a period of one hundred eighty (180) days or more, if you refused or failed the state's test(s). In addition, your commercial driving privilege in this state will be disqualified if you refused or failed the state's test(s) (operating a CMV or CDL holder) OR a test result of .04 or more while operating a commercial motor vehicle. To appeal the driver's license revocation/disqualification, the Department of Public Safety must receive your written request for an Administrative Hearing **within fifteen (15) days from the date you received notice of revocation or disqualification**. Submit your **written** request in person at 3600 N. MLK Ave. or by mail to Legal Division, Department of Public Safety, P.O. Box 11415, OKC, OK, 73136

Section 3

SERVING OF NOTICE: I personally hand-delivered a copy of the above containing the Notice of Revocation to the person named in Section 1 above.

Date served: **09/25/2017** Signature of officer: _____ Badge #: _____

In accordance with Title 12 O.S. Section 426, "I state under **penalty of perjury** under the laws of Oklahoma that the foregoing is true and correct."

Section 4

Date: **09/25/2017** Place (location when signed): _____ Signature of arresting officer: _____

| | | | |
|------------------------------|---------------------------------------|---------------------------------|------------------------------|
| NAME SMITH, JOSHUA | | AGENCY BOARD OF TESTS | |
| BADGE 0000000435 | AGENCY ADDRESS INTER-AGENCY | ZIP CODE 00000 | PHONE "4054252460" |

OTHER WITNESSES:

Section 5

| | | | | |
|---|------|-------|---------|-------|
| 1 | NAME | TITLE | ADDRESS | PHONE |
| 2 | NAME | TITLE | ADDRESS | PHONE |

No Temporary License Issued:
Test Below Legal Limit

Certificate of Analysis - Wet Bath

| | |
|-----------------------------------|--------------------------------|
| 05-05-17 | .020 |
| Date of Analysis | Labelled target value (g/210L) |
| 16400 | .0195 |
| Lot Number | Average test result (g/210L) |
| Jost Smith | |
| BOT Technician Name and Signature | |



GUTH LABORATORIES, INC.

1600 W. 10th St., P.O. Box 100, Erie, PA 17512-0100
 G, PA 17111-4511 • TELEPHONE: 717-564-5470

BRT Form 6.0

In accordance with BRT - 2.1.0, Maintenance, the above referenced simulator solution is suitable for use as an external reference in maintenance of the Intoxilyzer 8000.

CERTIFICATE OF ANALYSIS

INTOXILYZER 8000
 Instrument Initialization
 8133.12
 14:48 05/05/2017

Certified Alcohol Reference Solution for Simulator

Rec'd
 4/01/2017
 #46837

Random Samples of Lot Number **16400** of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on **December 13, 2016**, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain

ALPHA INSTRUMENT 2
 Intoxilyzer - Alcohol Analyzer
 Model 8000
 05/05/2017
 SN 80-004923
 14:56

Test g/210L Time **0.0241%** (w/vol) ethyl alcohol. The expiration date for this lot number is **December 12, 2018** at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of **0.020 g/210L +/- 3%**.

The alcohol and water used in this solution were free of test interfering substances.

| Test | g/210L | Time |
|-----------------|--------|-------|
| Air Blank | 0.000 | 14:57 |
| Cal Check | 0.020 | 14:58 |
| Air Blank | 0.000 | 14:58 |
| Cal Check | 0.020 | 14:59 |
| Air Blank | 0.000 | 14:59 |
| Cal Check | 0.019 | 15:00 |
| Air Blank | 0.000 | 15:01 |
| Cal Check | 0.020 | 15:01 |
| Air Blank | 0.000 | 15:02 |
| Cal Check | 0.020 | 15:03 |
| Air Blank | 0.000 | 15:03 |
| Cal Check | 0.020 | 15:04 |
| Air Blank | 0.000 | 15:04 |
| Cal Check | 0.019 | 15:05 |
| Air Blank | 0.000 | 15:06 |
| Cal Check | 0.019 | 15:06 |
| Air Blank | 0.000 | 15:07 |
| Cal Check | 0.019 | 15:08 |
| Air Blank | 0.000 | 15:08 |
| Cal Check | 0.019 | 15:09 |
| Air Blank | 0.000 | 15:09 |
| Cal Check Stats | | |
| Average | 0.0195 | |
| Std Dev | 0.0005 | |
| Rel Std Dev(%) | 2.7026 | |

16837

Ted L. Pauley
 Ted L. Pauley, President
 GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN11241503 whose values are traceable to NIST.
 All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.

| Certificate of Analysis – Wet Bath | |
|------------------------------------|--------------------------------|
| 3-8-2017 | .040 |
| Date of Analysis | Labelled target value (g/210L) |
| 16320 | .0409 |
| Lot Number | Average test result (g/210L) |
| Josh Smith <i>[Signature]</i> | |
| BOT Technician Name and Signature | |



GUTH LABORATORIES, INC.
 BURG, PA 17111-4511 • TELEPHONE: 717-564-5470

BRT Form 6.0

In accordance with BRT – 2.1.0, Maintenance, the above referenced simulator solution is suitable for use as an external reference in maintenance of the Intoxilyzer 8000.

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

REC'D
03/01/17
[Signature]
#16837

Random Samples of Lot Number **16320** of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on **October 25, 2016**, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain **0.0495%** (w/vol) ethyl alcohol. The expiration date for this lot number is **October 21, 2018** at 11:59 PM.

ALPHA INSTRUMENT 2
 Intoxilyzer - Alcohol Analyzer
 Model 8000 SN 80-004923
 03/08/2017 11:16

| Test | g/210L | Time |
|-----------------|--------|-------|
| Air Blank | 0.000 | 11:16 |
| Cal Check | 0.041 | 11:17 |
| Air Blank | 0.000 | 11:17 |
| Cal Check | 0.041 | 11:18 |
| Air Blank | 0.000 | 11:19 |
| Cal Check | 0.040 | 11:19 |
| Air Blank | 0.000 | 11:20 |
| Cal Check | 0.041 | 11:21 |
| Air Blank | 0.000 | 11:21 |
| Cal Check | 0.041 | 11:22 |
| Air Blank | 0.000 | 11:22 |
| Cal Check | 0.041 | 11:23 |
| Air Blank | 0.000 | 11:24 |
| Cal Check | 0.041 | 11:24 |
| Air Blank | 0.000 | 11:25 |
| Cal Check | 0.040 | 11:26 |
| Air Blank | 0.000 | 11:26 |
| Cal Check | 0.042 | 11:27 |
| Air Blank | 0.000 | 11:27 |
| Cal Check | 0.041 | 11:28 |
| Air Blank | 0.000 | 11:29 |
| Cal Check Stats | | |
| Average | 0.0409 | |
| Std Dev | 0.0005 | |
| Rel Std Dev(%) | 1.3878 | |

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of **0.040 g/210L +/- 3%**.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley
 Ted L. Pauley, President
 GUTH LABORATORIES, INC.

[Signature]
#16837

NIST Traceability:
 Testing was conducted using Cerilliant Reference Standard lot number FN1218501 whose values are traceable to NIST.
 All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.

| Certificate of Analysis – Wet Bath | |
|------------------------------------|--------------------------------|
| 06/15/2017 | .10 |
| Date of Analysis | Labelled target value (g/210L) |
| 17130 | .1015 |
| Lot Number | Average test result (g/210L) |
| J. Smith | |
| BOT Technician Name and Signature | |

BRT Form 6.0



GUTH LABORATORIES, INC.
 HARRISBURG, PA 17111-4511 • TELEPHONE: 717-564-5470

In accordance with BRT – 2.1.0, Maintenance, the above referenced simulator solution is suitable for use as an external reference in maintenance of the Intoxilyzer 8000.

CERTIFICATE OF ANALYSIS

Rec'd
 06/09/2017

ALPHA INSTRUMENT 2
 Intoxilyzer - Alcohol Analyzer
 Model 8000
 06/15/2017

SN 80-004923
 16:25

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 17130 ✓ of

| Test | g/210L | Time |
|-----------------|--------|-------|
| Air Blank | 0.000 | 16:26 |
| Cal Check | 0.102 | 16:26 |
| Air Blank | 0.000 | 16:27 |
| Cal Check | 0.102 | 16:28 |
| Air Blank | 0.000 | 16:28 |
| Cal Check | 0.101 | 16:29 |
| Air Blank | 0.000 | 16:29 |
| Cal Check | 0.102 | 16:30 |
| Air Blank | 0.000 | 16:31 |
| Cal Check | 0.101 | 16:31 |
| Air Blank | 0.000 | 16:32 |
| Cal Check | 0.101 | 16:33 |
| Air Blank | 0.000 | 16:33 |
| Cal Check | 0.102 | 16:34 |
| Air Blank | 0.000 | 16:34 |
| Cal Check | 0.101 | 16:35 |
| Air Blank | 0.000 | 16:36 |
| Cal Check | 0.102 | 16:36 |
| Air Blank | 0.000 | 16:37 |
| Cal Check | 0.101 | 16:38 |
| Air Blank | 0.000 | 16:38 |
| Cal Check Stats | | |
| Average | 0.1015 | |
| Std Dev | 0.0005 | |
| Rel Std Dev(%) | 0.5192 | |

Alcohol Reference Solution for Simulator were analyzed by gas chromatography on April 6, 2017, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1212% (w/vol) ethyl alcohol. The expiration date for this lot number is April 3, 2019 at 11:59 PM. ✓

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%. ✓

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President
 GUTH LABORATORIES, INC.

16837
 Operator's Signature

NIST Traceability:
 Testing was conducted using Cerilliant Reference Standard lot number FN08051301 whose values are traceable to NIST.
 All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.

| Certificate of Analysis – Wet Bath | |
|------------------------------------|--------------------------------|
| 06-16-17 | .20 |
| Date of Analysis | Labelled target value (g/210L) |
| 17090 | .2045 |
| Lot Number | Average test result (g/210L) |
| J. Smith | |
| BOT Technician Name and Signature | |



LABORATORIES, INC.
 HARRISBURG, PA 17111-4511 • TELEPHONE: 717-564-5470

BRT Form 6.0

In accordance with BRT – 2.1.0, Maintenance, the above referenced simulator solution is suitable for use as an external reference in maintenance of the Intoxilyzer 8000.

REC'D
 06/09/17
 16837

CERTIFICATE OF ANALYSIS

ALPHA INSTRUMENT 2
 Intoxilyzer - Alcohol Analyzer
 Model 8000 SN 80-004923
 06/16/2017 10:52

Certified Alcohol Reference Solution for Simulator

| Test | g/210L | Time |
|-----------------|--------|-------|
| Air Blank | 0.000 | 10:53 |
| Cal Check | 0.204 | 10:53 |
| Air Blank | 0.000 | 10:54 |
| Cal Check | 0.205 | 10:55 |
| Air Blank | 0.000 | 10:55 |
| Cal Check | 0.205 | 10:56 |
| Air Blank | 0.000 | 10:57 |
| Cal Check | 0.205 | 10:57 |
| Air Blank | 0.000 | 10:58 |
| Cal Check | 0.205 | 10:59 |
| Air Blank | 0.000 | 10:59 |
| Cal Check | 0.205 | 11:00 |
| Air Blank | 0.000 | 11:00 |
| Cal Check | 0.204 | 11:01 |
| Air Blank | 0.000 | 11:02 |
| Cal Check | 0.204 | 11:02 |
| Air Blank | 0.000 | 11:03 |
| Cal Check | 0.204 | 11:04 |
| Air Blank | 0.000 | 11:04 |
| Cal Check | 0.204 | 11:05 |
| Air Blank | 0.000 | 11:05 |
| Cal Check | 0.000 | 11:05 |
| Cal Check Stats | | |
| Average | 0.2045 | |
| Std Dev | 0.0005 | |
| Rel. Std Dev(%) | 0.2577 | |

Random Samples of Lot Number 17090 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on February 27, 2017, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.2416% (w/vol) ethyl alcohol. The expiration date for this lot number is February 24, 2019 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.200 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley
 Ted L. Pauley, President
 GUTH LABORATORIES, INC.

J. Smith
 Operator's Signature

NIST Traceability:
 Testing was conducted using Cerilliant Reference Standard lot number FN08101505 whose values are traceable to NIST.
 All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.