

STATE OF OKLAHOMA BOARD OF TESTS FOR ALCOHOL AND DRUG INFLUENCE

Dr. Jarrad Wagner, Ph.D. F-ABFT Chairman

Post Office Box 36307 Oklahoma City, Oklahoma 73136-2307 Phone: (405) 425-2460 Fax: (405) 425-2490 www.bot.ok.gov

Joshua Smith Director

ATTESTATION

I, the undersigned <u>Joshua Smith</u> (Director/Records Custodian), attest under oath pursuant to 12 O.S. §2902 (11)(a) this record was made, at or near the time of the occurrence of the matters set forth by or from information transmitted by a person having knowledge of those matters; was kept in the course of the regularly conducted business activity of the Board; and was made pursuant to the regularly conducted activity. The duplicate record provided is listed below:

1) The attached "Service Overview" is a true and accurate copy of maintenance performed on the Intoxilyzer 8000, serial number 80-004202, in compliance with the agency's Maintenance Policy and Procedures.

This packet consists of 13 pages.

signed _____

Name of Position: Director/Records Custodian

Date of Attestation: 05/23/2022

Service Overview

3600 N. Martin Luther King Bldg #9 Oklahoma City, OK 73111 405-425-2460

BENCH CHECK DATE: <u>08-19-2019</u>

SERIAL: <u>80-004202</u>

DOCUMENTATION WITHIN SERVICE PACKET:

- BOT Certificate of Calibration and Operation
- Bench Check
- Bench Check Technician's Report
- BOT Dry Gas Certificate of Analysis
- Mock Subject Test
- BOT Certificates of Analysis on Certified Solutions (4)



STATE OF OKLAHOMA BOARD OF TESTS FOR ALCOHOL AND DRUG INFLUENCE

Administrative Office at:
Board of Tests for Alcohol and Drug Influence
Post Office Box 36307
Oklahoma City, Oklahoma 73136-2307

Certificate of Calibration and Operation

This is to certify that the calibration of INTOXILYZER 8000, serial number 80-004202, manufactured by CMI Inc., a subsidiary of MPD, Inc. of Owensboro, Kentucky, was tested and found to be in compliance with National Highway Traffic Safety Administration Standard for Devices to Measure Breath Alcohol (F.R., No. 179 48705-48710) Sept. 17, 1993) and with International Recommendation OIML R 126 Edition 1998 (E). Calibration solutions are traceable to NIST standard material 1828.

In addition the operational procedures and functions of this Intoxilyzer have also been tested and found to be in compliance with the Oklahoma Board of Tests for Alcohol and Drug Influence - Rules and Regulations OAC Title 40: 25-1-2 and 30-1-3 and is hereby certified for use as an evidential breath testing device within this state.

CERTIFICATION TYPE	DATE
Manufacturer Calibration	06-03-09
Board of Tests Bench Check	07-06-09
Board of Tests Bench Check	10-07-11
Board of Tests Bench Check	08-20-13
Manufacturer Calibration	09-11-15
Board of Tests Bench Check	09-24-15
Board of Tests Bench Check	08-02-17
Board of Tests Bench Check	09-06-17
Board of Tests Bench Check	08-19-19

OEDTIEIO ATION TION



Presentation of this form certifies that the Intoxilyzer, SN: 80-004202, functioned properly at the time of the breath test and hereby certifies the breath test result as valid.

Date:

Re: Request for Log of Test on Intoxilyzer 8000

Name:

Dear Sir:

The Log of Test information from a breath test conducted on a Intoxilyzer 8000 is printed in Section 2 of the Affidavit and is no longer retained as a separate form as with the Intoxilyzer 5000. The 8000 uses a nitrogen/ethanol dry gas for the external control test. The dry gas cylinder has the capacity to perform approximately 325 tests and does not require maintenance until its expiration date or can no longer provide the required pressure to perform a control test. The cylinder has an expiration date of two years from date filled at the manufacturer and is programmed in the 8000 upon installation, both dates are printed in Section 2 of the Affidavit. The Intoxilyzer is programmed to disable itself when the date of expiration is reached or the pressure monitor indicates 25 lbs psi.

Due to the reliability and stability of dry gas, maintenance on the 8000 is not required every 30 days or 25 tests, therefore maintenance is no longer performed by officers in the field. The Board of Tests for Alcohol and Drug Influence certifies each Intoxilyzer 8000 for field use by performing a bench check. The bench check is designed to check the operation and procedures programmed in the 8000 to ensure it is functioning properly for use in Oklahoma. A bench check is performed every time a new dry gas cylinder is installed or the instrument has returned from the manufacturer for repair. Once the instrument passes the bench check, all that is required to set it up for field use is to plug it in and turn it on and does not require a permit to set up. The Board of Tests has the ability to monitor each instrument once it is set up in the field.

Each Intoxilyzer has a database of valid permit numbers that it checks prior to allowing a breath test to be performed. The database is updated on a regular basis, if a permit number is invalid that officer will not be able to conduct a breath test on the 8000. If the 8000 prints an affidavit the breath test operator listed in Section 2 had a valid permit at the time of the test and the breath test is valid.

If you have any further questions please feel free to call me.

Sincerely

J. Robert Blakeburn (OBA11059) State Director of Tests for Alcohol and Drug Influence

O. pohr &Blokelune

STATE OF OKLAHOMA - BOARD OF TESTS FOR ALCOHOL AND DRUG INFLUENCE CMI INTOXILYZER BENCH CHECK PROCEDURE REPORT

NCH CHECK DATE 08/19/2019	TIME PROCEDURE BEGAN		80-00420	2	7.1
RUMENT TYPE ntoxilyzer	INSTRUMEN	IT MODEL		80-00	
iicoxiiyzei	1-80	TEST RE	SULTS	80-00	4202
		TEGTICE	00210		
Diagnosti					
PASS					
Mouth Alcoho	ol Test				
PASS					
RFI Tes	t				
PASS		2			and the
Abort Te	st	Duj	olicate Breath	Sampling Re	esults
PASS		Value	Test 1	Test 2	Control Test
Improper Samp	ole Test				40.00
PASS		.02	0.018	0.018	0.018
Interferent Dete	ect Test	.04	0.034	0.033	0.035
PASS		10	0 004	0 003	0.005
No Sample Giv	en Test	.10	0.094	0.093	0.095
PASS		.20	0.196	0.196	0.199
Insufficient Sam	inle Test		PA	SS	A
PASS	ipio resc				
Air Blank Contami					
PASS	nation rest				
Range Exceed	ed Test				
PASS					
		CERTIFYING T	ECHNICIAN		
swear under penalty	y of perjury that in	regards to the	above listed i	instrument, I	have complied
he Intoxliyzer Bencl	h Checking Proced	ure approved	by the State o	f Oklahoma.	
TURE				DATE	
((1)				0	8/19/2019
				Ĭ	-traffer
ITH, JOSHUA	A n			PERMIT#	016837



3600 N. Martin Luther King Bldg #9 Oklahoma City, OK 73111 405-425-2460

BENCH CHECK TECHNICIAN'S REPORT

20		Serial #	1.00
00 AS CANISTER		80-0042	202
T#		EXP Date	
7919080A2	# 008	08/06	12021
EFERENCE	11 000		100
	Simula	ator Model	
Guth 2100	Guth 2100	Guth 2100	Guth 2100
		erial #	
DR3591	DR3753	DR3594	DR3754
.02	.04	centration .10	.20
.02		.10 .OT #	.20
18020	17410	18070	17340
	Manufa	ctured Date	
01-09-2018	12-06-2017	02-26-2018	10-09-2017
	Expira	ation Date	
01-09-2020	12-06-2019	02-26-2020	10-09-2019
	Solution Co	mmission Date	
08-01-19	08-01-19	08-05-19	08-05-19
EASON FOR BE POST REPAIR	CYLINDER REP	LACEMENT	
	CYLINDER REP	LACEMENT E MAINTENANCE	^
POST REPAIR D TROUBLESHOOT	CYLINDER REPI	LACEMENT E MAINTENANCE	
POST REPAIR DE TROUBLESHOOT DIMMENTS	CYLINDER REPI	LACEMENT E MAINTENANCE	
POST REPAIR DE TROUBLESHOOT DIMMENTS	CYLINDER REPI	LACEMENT E MAINTENANCE	
POST REPAIR DE TROUBLESHOOT DIMMENTS	CYLINDER REPI	LACEMENT E MAINTENANCE	

Printed Name

g/210L

0.000

0.080 0.000

0.080

0.000

0.080

0.000

0.080

0.000

0.080

0.000

0.081

0.000

0.080

0.000

0.080

0.000

0.080

0.000

0.080

0.000

0.0801

0.0003

0.3947

SN 80-002591

11:17

Time

11:18

11:18

11:18

11:19

11:19

11:20

11:20

11:21

11:21

11:21

11:22

11:22

11:23

11:23

11:24

11:24

11:24

11:25

11:25

11:26

11:26

ALPHA INSTRUMENT

Model 8000

08/12/2019

Test

Air Blank

Cal Check

Air Blank Cal Check Stats Average

Std Deu

Rel Std Deu(%)

Intoxilyzer - Alcohol Analyzer

Certificate	of Analysis – Dry Gas	1
08-12-19	,080	
Date of Analysis	Labelled target value (g/210L)	1
#008	- ,080	
Cylinder#	Average test result	= ILM (:)
Josef Smith	2	ILIVIO
BOT Technician Name and Signatur	e A	BRT Form 5.0
	and Procedure Statement BRT-2,1.0, Mo	

7 Eastgate Dr. • P.O. Box 790 • Jacksonville, IL 62651-0790 217-245-2183 • Fax: 217-243-7634 • www.ilmoproducts.com

Certificate of Analysis

Certificate ID:

12204

Part #:

BAC105L080T

Cylinder Size:

1.05L

Lot Number:

17919080A2

Expiration:

8/5/2021

0.080 BAC (For the calibration of instruments used to determine breath alcohol concentration)

Contents:

Component:

105 Liters @ 1000 psig 70°F (21°C)

Analytical

Reported

Concentration:

Accuracy

(U, k=2):

Analytical Method:

Ethanol

208 ppm

+/-0.002 BAC(G/210L)

Nitrogen

[5.2 ppm]

Distributed by:

CMI Inc. 316 East Ninth Stree Owensboro, KY 423 Phone 866-835-0690 www.alcoholtest.cor

Operator's Signature

*Traceable to: Certified Reference Material - 262.4 µmol/mol Ethanol in Nitrogen - Serial No. GN0015026 Lot No. 050319E11

Store in dry area, away from sources of heat, ignition and direct sunlight. Do not allow storage area to exceed 52 °C (125 °F).

Osh Uring Specialty Gas Lab Tech

Issuance Date

07-15-19

Accreditation #61895

The calibration results within this certificate were obtained using equipment and standards capable of producing analytical results traceable to NIST, and apply only to the items contained on this certificate. ILMO Products Company makes no warranty or representation as to the suitability of the use of any information provided for any particular purpose. The information use is at the sole discretion and risk of the user. Liability shall be limited to established replacement cost of this material or service.

8/13

OFFICER'S AFFIDAVIT AND NOTICE OF

	R	EVOCAT	rion/di	SQUA	LIFI	CATIO	N			Section
ARREST DATE 08/20/2019		ARREST TIME		COUNTY #		82	CITATIO			
ARREST LOCATION		10000		133	CITY		1110	COUNT		
MOCK SUBJECT NAME					MOC	OF BIRTH	SE	X RACE	HEIGHT	WEIGHT
MOCK, SUBJE	ECT TEST					3/09/19	51 N	l l	510	165
ADDRESS 3600 MARTIN	LUTHER	KING A	OKLA	AHOMA C	'ITY		STATE		ZIP CODI	
DRIVER LICENSE # T083454871				06/30		STATE OK	LASS	RESTR	ICTIONS	ENDORSEMENT M
VEHICLE MAKE		MODEL		106/30		AG#	<u>A</u>	STATE	CMV	PLACARD
MOCK		MOCK				MOCK		OK	N	N
On the above date, time, and loci- highway, street, turnpike, public p influence of alcohol and/or other i (Describe driving behavior or circo MOCK SUBJECT	earking lot, or other pub ntoxicating substances umstances): TEST	lic place, or upon a private	road, street, alley or la	ne which provides	access to one	or more single or n	nulti-family o	dwelling within	this state wi	a public road, hile under the
Describe person's condition (odor	•		CHED TAXOUR	. . .						
ROUTINE MAINT	ENANCE, N	EW GAS CANI	STER INSTA	ЯLL						
		THE PERSON V	VAS INFORMED OF T	HE IMPLIED CON	NSENT TEST I	REQUEST				
BREATH-ALCOHOL ANALYSIS	RECORD, REPORT C	F THE PERSON NAMED	IN SECTION 1 ABOVE	, and LOG OF TE	ST AND MAIN	TENANCE RECOR	D.		****	Section
EQUIPMENT I - 8000		}	NUMBER : 004202	SPECIALIST BOARD	OF TE	יפידפ				
STANDARD	······································		TARGET VALUE		PERMIT NUM	1BER	*			
Dry		···-	0.080		1683	7 DER INSTALLED	Ir	DATE CYLIND	ER EXPIRE	\$
ILMO			179190)80A2		9/2019		08/05	/202	1
OBSERVATION START TEST 0940 08	DATE 3/20/2019	MOUTHPIECE G	A 15-minut	e (or greater)) deprivati	on/observatio	n			
RESULT TABLE	7 20 7 20 1 3	, , ,		as observed page time of tes		d continuing			THE S	TAR
Air Blank Subject Test Breath Volume Air Blank Wait Air Blank Subject Test	0.00 0.00 1.402 LITER 0.00	09:59 10:01 10:01 10:02	_	e with the dure of the	applical	ble operatii	1	OHEAT SEAL	100	
Breath Volume Air Blank	1.914 LITER 0.00	10:02	SIGNATURE OF	OI ENATOR						
Cal Check	0.08	10:03	NAME SMITH, JO	OSHUA					Test Re	
Air Blank	0.00	10:03	BADGE#		PERMIT#	016000		В	rAC g/	210L
			0000004 AGENCY	435	0000	016837			0.0	0 (
			BOARD OF	TESTS						
EFFECTIVE THIRTY (30) DAY lays or more, if you refused or operating a CMV or CDL holde Safety must receive your writter The approved hearing reques Division, Department of Public to I fine of not more than Five Hu-	failed the state's test yr) OR a test result of n request for an admi st form is available a Safety, P.O. Box 114 ndred Dollars (\$500.0 er evidence of driving	(s). In addition, your con .04 or more while operal inistrative hearing on the st https://www.ok.gov/d15, OKC, OK 73136. It s .00), or by both such fine privilege is in the custod	nmercial driving privile ting a commercial mo to Department approv ps/documents/Reql hall be a misdemean and imprisonment, for y of a law enforceme	ege in this state votor vehicle. To a ved form, within the Hearing.pdf. Sulor punishable by a rany person to a nt officer or the E	vill be disqual ppeal the driv fifteen (15) da bmit your hea imprisonmen pply for a ren Department.	ified if you refused yer's license revoca ys from the date y ring request form t for not less than ewal or a replacen	or failed the stion/disquerou receive in person a seven (7) conent licens	ne state's test alification, the d notice of re at 3600 N. ML days nor more e to operate a	t(s) Departme vocation or K Ave. or to the than six (6 motor veh	r disqualification by mail to Legal 3) months, or by nicle while the
SERVING OF NOTICE: Date served: 08/20				ning the Notice	e of Revoca	tion to the pers		ed in Section Badge #:_	on 1 abov	⁄e.
n accordance with Title 1	2 O.S. Section 42	26, "I state under pe i	nalty of perjury (under the laws	of Oklaho	ma that the fore	egoing is	true and c	orrect."	Section
Date: 08/20/201	9 Place (location	on when signed):		AGENCY	Signatur	e of arresting o	fficer:			
SMITH, JOS	HUA			BOARD	OF T	ESTS				
BADGE 0000000435	AGENCY ADDRESS INTER-A	GENCY				ZIP CODE 00000	PHONE 4 C)54252	2460'	ı
THER WITNESSES:										Section
NAME			TITLE	ADDRESS			PHO	ONE		

ADDRESS

PHONE

TITLE

OFFICER'S AFFIDAVIT AND NOTICE OF REVOCATION/DISQUALIFICATION

			_							Section I
ARREST DATE 08/20/2019	ARREST TIME 0900		COUNTY#	CITY 8.			TATION # MOCK			
ARREST LOCATION MOCK	<u> </u>		CIT	OCK				COUNTY		
SUBJECT NAME				DATE OF			SEX	RACE	HEIGHT	WEIGHT
MOCK, SUBJECT TEST				08/	09/1		M	W	510	165
3600 MARTIN LUTHER	KING A	OKLAHO					OK		ZIP CODE 731	33
T083454871		C	PIRATION DATE 06/30/20)19	STATE OK	CLASS A		RESTRIC		M
VEHICLE MAKE MOCK	MOCK			TAG M	# OCK		STA	K	CMV F	N
On the above date, time, and location, the above name- highway, street, turnpike, public parking lot, or other put- influence of alcohol and/or other intoxicating substances (Describe driving behavior or circumstances): MOCK SUBJECT TEST Describe person's condition (odor, actions, etc.):	olic place, or upon a private	1 had reasonable grounds to e road, street, alley or lane wh	o believe the person nich provides access	was driving to one or	g or in actua more single	al physical o	control of a	motor vel g within th	hicle upon a his state whi	public road, ile under the
ROUTINE MAINTENANCE, N	EW GAS CANI	STER INSTALL	1							
	THE PERSON V	WAS INFORMED OF THE IN	MPLIED CONSENT	TEST REQ	UEST					
BREATH-ALCOHOL ANALYSIS RECORD, REPORT C				MAINTEI	NANCE RE	CORD.				Section 2
I-8000		-004202 BG		TES						
STANDARD Dry	•	TARGET VALUE 0.080		IT NUMBE	R					
MANUFACTURER ILMO		17919080	DATE	CYLINDER	2019				REXPIRES	
OBSERVATION START TEST DATE	MOUTHPIECE	A 15-minute (o		·			U8	705	/ 202	<u> </u>
0940 08/20/2019 RESULT TABLE	9 G	}	bserved prior					_	- F 00	
Test g/210L	Time	to the ti	me of test adı	ninistra	tion.			OF.	***	ATE
Air Blank 0.00 Subject Test 0.00 Breath Volume 1.402 LITER Air Blank 0.00 Wait Air Blank 0.00 Subject Test 0.00	09:59 10:01 10:01 10:02	I administered compliance w procedur	vith the app re of the Bo	licable	e opera	ating	6		1907	
Breath Volume 1.914 LITER Air Blank 0.00	10:02									
Cal Check 0.08 Air Blank 0.00	10:03 10:03	NAME SMITH, JOSH	IUA						Test Res	
All Blank 0.00	10:03	BADGE # 00000043	5 0.0		L6837	7			AC g/2	
		AGENCY BOARD OF T					1 L	(0.0	0
EFFECTIVE THIRTY (30) DAYS FROM DATE SER days or more, if you refused or failed the state's test (operating a CMV or CDL holder) OR a test result of Safety must receive your written request for an adm The approved hearing request form is available Division, Department of Public Safety, P.O. Box 114 a fine of not more than Five Hundred Dollars (\$500.)	(s). In addition, your cor .04 or more while opera inistrative hearing on that https://www.ok.gov/c15, OKC, OK 73136. It s	ation is given that your driv nmercial driving privilege in ting a commercial motor ve Department approved for dps/documents/ReqHeari shall be a misdemeanor pu and imprisonment, for any	ver's license is revo n this state will be o ehicle. To appeal i orm, within fifteen ing.pdf. Submit yo nishable by imprisi person to apply fo	disqualified the driver's (15) days our hearing conment for r a renewa	d if you refu s license re from the da g request for not less the	used or fail evocation/d ate you rec orm in pers han seven	ed the sta lisqualifica ceived noti son at 360 (7) days n	te's test(tion, the ice of rev 0 N. MLk nor more	s) Departmen ocation or (Ave. or by than six (6)	disqualification. y mail to Legal) months, or by
person's license, permit, or other evidence of driving SERVING OF NOTICE: I personally hand		,			n to tho	norcon n	amad in	Soction	n 1 abay	
Date served: 08/20/2019		-	the Notice of K	evocatio	n to the p	person n		ge #:	niabove	∃ .
In accordance with Title 12 O.S. Section 42			er the laws of O	klahoma	that the	foregoin			rrect."	Section 4
Date: 08/20/2019 Place (location						ng officer	•			
SMITH, JOSHUA		1	ENCY BOARD OF	TE	STS					
BADGE O 0 0 0 0 0 0 4 3 5 INTER-A	GENCY				ZIP CODE) () PHO		252	460"	





BRT Form 6.0

In accordance with BRT – 2.1.0, Maintenance, the above referenced simulator solution is suitable for use as an external reference in maintenance of the intoxilyzer 8000.

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 18020 of

Alcohol Reference Solution for Simulator were analyzed by
gas chromatography on January 11, 2018, using a Perkin Elmer Gas
Chromatograph Autosystem XL S/N: 610N9030209, and found to contain
0.0238% (w/vol) ethyl alcohol. The expiration date for this lot
number is January 9, 2020 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.020 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Operator's Signature

_PHA INSTRUMENT

odel 8000

3/26/2018

ir Blank

al Check

ir Blank

al Check

ir Blank

il Check

r Blank

il Check

r Blank

il Check

r Blank

1 Check

r Blank

I Check

r Blank

1 Check

Blank

I Check

Blank

1 Check

Blank

d Deu

Check Stats

1 Std Deu(%)

ntoxilyzer - Alcohol Analyzer

g/210L

.0.000

0.020

0.000

0.020

0.020

0.000

0.020

0.000

0.020

0.000

0.020

0,000

0.021

0.000

0.021

0.000

0.021

0.000

0.021

0.000

0.0204

0.0005

2.5312

SN 80-002591

15:36

Time

15:36

15:37

15:37

15:38

15:39

15:39

15:40 15:41

15:41

15:42

15:42

15:43

15:44

15:44

15:45

15:46

15:46

15:47

15:47

15:48

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN08031603 whose values are traceable to NIST.



Certificate of Analysis – Wet Bath
11/13
09-26-18
Date of Analysis
Labelled target value (g/210L)

9.0396



BRT Form 6.0

CERTIFICATE OF

In accordance with BRT – 2.1.0, Maintenance, the above referenced simulator solution is A suitable for use as an external reference in maintenance of the intoxilyzer 8000.

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 17410 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on December 7, 2017, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.0481% (w/vol) ethyl alcohol. The expiration date for this lot number is December 6, 2019 at 11:59 PM.

BOT Technician Name and Signature

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.040 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Operator's Signature

ALPHR INSTRUMENT Intoxilyzer - Alcohol Analyzer

g/210L

0.000

0.040

0.000

0.039

0.000

0.039

0.000

0.040

0.000

0.040

0.000

0.039

0.000

0.040

0.000

0.040

0.000

0.039

0,000

0,0005

1.3040

SN 80-002591

15:54

Time

15:54

15:55

15:56

15:56

15:57

15:58

15:58

15:59

15:59

16:00

16:01

16:01

16:02

16:03

16:03

16:04

16:04

16:05

16:06

16:06

Model 8000

09/26/2018

Test

Air Blank

Cal Check

Air Blank


Cal Check Stats Average Std Dev

Rel Std Deu(%)

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN12181501 whose values are traceable to NIST.

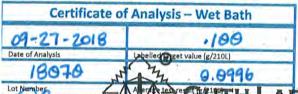

OUTH LABORATORIES, INC.

590 NORTH 67th STREET

HARRISBURG, PA 17111-4511
 TELEPHONE: 717-564-5470

BRT Form 6.0

In actor ance with BRT – 2.1.0, *Maintenance*, the above referenced simulator solution is suitable for use as an external reference in maintenance of the intoxilyzer 8000.

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 18070 of

Alcohol Reference Solution for Simulator were analyzed by
gas chromatography on February 28, 2018, using a Perkin Elmer Gas
Chromatograph Autosystem XL S/N: 610N9030209, and found to contain
0.1219% (w/vol) ethyl alcohol. The expiration date for this lot
number is February 26, 2020 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Operator's Signature

ALPHA INSTRUMENT Intoxilyzer - Ali

Model 8000

09/27/2018

Test

air Blank

Cal Check

Air Blank

Cal Check

air Blank

Cal Check

ir Blank

lat Check

Wir Blank

lal Check

lir Blank

al Check

ir Blank

al Check

tir Blank

al Check

ir Blank

al Check

ir Blank

al Check

ir Blank

Std Dev Rel Std Dev(%)

al Check Stats Average

Alcohol Analyzer

0.000

0.100

0.000

0,100

0.000

0.099

0.000

0.100

0.000

0.100

0,000

0.100

0.000

0,099

0.000

0.099

0.000

0.099

0.100

0.000

0.0996

0.5184

SN 80-002591

08:28

Tine

08:28

18:29

08:29

08:30

08:31

08:31

08:32

08:33

08:33

08:34

08:34

08:35

08:36

08:36 08:37

08:38

08:38

08:39


08:39

08:40

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN04271602 whose values are traceable to NIST.

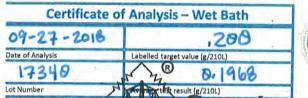

BOT Technician Name and Signature

LPHA INSTRUMENT

0008 lsbc

r Blank

il Check

r Blank

il Check

r Blank

1 Check

r Blank

1 Check

r Blank

1 Check

r Blank

I Check

Blank

1 Check

1 Check

Blank

Check

Blank

Check

Blank

ierage d Deu

Check Stats

1 Std Deu(%)

Blank

htoxilyzer - Alcohol Analyzer

g/210L

0.196

0.000

1 196

0.000

0.197

0,000

0.197

0.000

0.197

0.000

0.197

0.000

0.197

0.000

0.197

0.000

0.197

0.000

0.197

0.000

0.1968

0.0004

0.2142

SN 80-002591

15:27

Time

15:27

15:28

15:29

15:29

15:30

15:31

15:31

15:32

15:33

15:33

15:34

15:35

15:36

15:36

15:37

15:38

15:38

15:39

15:40

GUTH LABORATORIES, INC.

HARRISBURG, PA 17111- 4511 TELEPHONE: 717-564-5470

In accordance with BRT -2.1.0, Majorenance, the above referenced simulator solution is suitable for use as an external reference in maintenance of the Intoxilyzer 8000.

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 17340 of
Alcohol Reference Solution for Simulator were analyzed by
gas chromatography on October 10, 2017, using a Perkin Elmer Gas
Chromatograph Autosystem XL S/N: 610N9030209, and found to contain
0.2410% (w/vol) ethyl alcohol. The expiration date for this lot
number is October 9, 2019 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.200 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Operator's Signature

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN08101505 whose values are traceable to NIST.