

Dr. Jarrad Wagner, Ph.D. F-ABFT Chairman

Post Office Box 36307 Oklahoma City, Oklahoma 73136-2307 Phone: (405) 425-2460 Fax: (405) 425-2490 www.bot.ok.gov

Joshua Smith Director

ATTESTATION

I, the undersigned *Joshua Smith* (Director/Records Custodian), attest under oath pursuant to 12 O.S. §2902 (11)(a) this record was made, at or near the time of the occurrence of the matters set forth by or from information transmitted by a person having knowledge of those matters; was kept in the course of the regularly conducted business activity of the Board; and was made pursuant to the regularly conducted activity. The duplicate record provided is listed below:

1) The attached "Service Overview" is a true and accurate copy of maintenance performed on the Intoxilyzer 8000, serial number 80-005080, in compliance with the agency's Maintenance Policy and Procedures.

Signed

Name of Position: Director/Records Custodian

Date of Attestation: 01/05/2023

Service Overview



3600 N. Martin Luther King Bldg #9 Oklahoma City, OK 73111 405-425-2460

BENCH CHECK DATE: 01/05/2023

SERIAL: 80-005080

DOCUMENTATION WITHIN SERVICE PACKET:

- BOT Certificate of Maintenance, Calibration and Operation
- Bench Check
- BOT Dry Gas Certificate of Analysis
- Ilmo Specialty Gases Certificate of Analysis
- Mock Subject Test
- BOT Certificate of Analysis on Certified Solutions (4)
- Guth Laboratories Certificate of Analysis (4)



Post Office Box 36307 Oklahoma City, Oklahoma 73136-2307 Phone: (405) 425-2460 Fax: (405) 425-2490 www.bot.ok.gov

CERTIFICATE of MAINTENANCE, CALIBRATION, and OPERATION

This certifies the Intoxilyzer 8000 referenced below, manufactured by CMI, Inc., a subsidiary of MPD, Inc. of Owensboro, Kentucky, is maintained by the Board of Tests for Alcohol and Drug Influence in accordance with 40 O.A.C. 30-1-3. The instrument reported below is hereby deemed to be in compliance with Board rules via linearity accuracy examination and operational/functional procedure evaluation also referred to as a "Bench Check". The Intoxilyzer 8000 appears on the National Highway Traffic Safety Administration's Conforming Products List of Evidential Breath Alcohol Measurement Devices (82 FR 50940 *et seq*) as an approved device. In accordance with 40 O.A.C. 25-1-2, this instrument is approved and certified for use in the State of Oklahoma as an automated evidential breath-alcohol measurement device by the Board of Tests pursuant to 47 O.S. § 759.

All analyses conducted by this instrument are considered precise, valid, and in accordance with 40 O.A.C. 30-1-3(b) Operating Procedure(s). A printed breath test operator affidavit containing the seal of the Board by this instrument confirms a calibration check successfully verified the instrument's calibration at the time of test. The printed breath test affidavit also serves as proof that safeguards in place to protect the breath samples, including but not limited to Ambient Air Interference, Radio Frequency Interference, Residual Mouth Alcohol, Deficient Samples, etc. were not triggered and the test is a true and accurate scientific measurement of the person's alcohol concentration.

- Maintenance was performed on the Intoxilyzer 8000, Serial No. 80-005080 on 01/05/2023 in accordance with 40 O.A.C. 30-1-3 and internal BOT maintenance policy and procedures.
- A dry gas canister appearing on the National Highway Traffic Safety Administration's Conforming Products List of Calibrating Units for Breath Alcohol Testers (77 FR 64588 et seq) and approved in accordance with 40 O.A.C. 25-1-3 and BOT maintenance policy was installed.
 - Manufacturer: ILMO Unit: 208 ppm / 105 L
 - Cylinder Lot No. Canister No.: <u>2642080A1-058</u>
 BOT analysis average test result: 0.081
 - Cylinder expiration date: 11/05/2024

3. In accordance with 40 O.A.C. 25-1-2.1, the approved reference standards and equipment were used:

Simulator Model			X and the same series
Guth 2100	Guth 2100	Guth 2100	Guth 2100
Serial #			
DR2834	DR3694	DR2455	DR3470
Concentration			
.02	.04	.10	.20
LOT#			Town Assess
21340	21410	22080	22050
Manufactured Date			
8/24/2021	9/30/2021	3/7/2022	2/7/2022
Expiration Date			
8/24/2023	9/30/2023	3/7/2024	2/7/2024
Solution Commission Date			
12/13/2022	12/13/2022	12/16/2022	12/16/2022

Completed	by	BOT	Personnel:
-----------	----	-----	------------

Christopher Pape

Breath-Alcohol Testing Program Administrator

Title

Certified by:

Signature

Signature/Nat

Joshua Smith Name State Director Title

STATE OF OKLAHOMA - BOARD OF TESTS FOR ALCOHOL AND DRUG INFLUENCE CMI INTOXILYZER BENCH CHECK PROCEDURE REPORT

/05/2023	100000000000000000000000000000000000000	EDURE BEGAN		RUMENT LOCATION 0 - 005080)	
ument type ntoxilyzer		INSTRUMENT MODEL I - 8000			80-005	
			TRES	JLTS		
Diagnosti	cs					
PASS	77					
Mouth Alcoho	ol Test					
PASS	117				Į.	
RFI Test						
PASS						
Abort Te	et		Dupl	icate Breath	Sampling Re	sults
PASS	51	Val	ue	Test 1	Test 2	Control Test
Improper Samp	ole Test		15		r con e la ruetari e	
PASS		.0	2	0.018	0.017	0.018
Interferent Dete	ect Test	.0	4	0.036	0.036	0.038
PASS		1	.0	0.098	0.098	0.099
No Sample Giv				0.195	1 5 1 5 5 5 5 5 5 5 5 5 5 5 5 5 5 5 5 5	6 7615
PASS		.2	U		0.194	0.196
Insufficient Sam	ple Test			PA	SS	
PASS						
Air Blank Contami	nation Tes	t				
PASS						
Range Exceed	ed Test					
PASS						



Dr. Jarrad Wagner, Ph.D., F-ABFT Chairman

Post Office Box 36307 Oklahoma City, Oklahoma 73136-2307 Phone: (405) 425-2460 Fax: (405) 425-2490 www.bot.ok.gov

Joshua Smith Director

Certificate of Analysis - Dry Gas

ALPHA INSTRUMENT

Intoxilyzer - Alcohol Analyzer

Model 8000

11/14/2022

SN 80-002591

08:37

Test	g/210L	Time
Air Blank	0.000	08:37
Cal Check	0.081	08:37
Air Blank	0.000	08:38
Cal Check	0.081	08:38
Air Blank	0.000	08:39
Cal Check	0.081	08:39
Air Blank	0.000	08:40
Cal Check	0.081	08:40
Air Blank	0.000	08:40
Cal Check	0.081	08:41
Air Blank	0.000	08:41
Cal Check	0.081	08:42
Air Blank	0.000	08:42
Cal Check	0.081	08:42
Air Blank	0.000	08:43
Cal Check	0.081	08:43
Air Blank	0.000	08:44
Cal Check	0.081	08:44
Air Blank	0.000	08:45
Cal Check	0.080	08:45
Air Blank	0.000	08:46
Cal Check Stats		
Average	0.0809	
Std Dev	0.0003	
Rel Std Dev(%)	0.3908	

Known Breat	h-Alcohol Equivalent (g/210L) Ratio of Nitrogen and 0.08 g/210L = 208 PPM ÷ 2605*	Ethanol Gas (208 ppm/105 L)
Manufacturer: ILMO	Manufacturer Cert. No.: 15117	Cylinder Size: 105L
Lot No.: 26422080A1	Cylinder No.: 058	Expiration Date: 11/05/2024
prescribed by the State Director of Tests p calibrating unit in accordance with 47 O.S	oursuant to 40 OAC 30-1-3. The above NIST traceable . 759, 40 OAC 25-1-3, and appears on the current confe	100 and was performed in accordance with the procedure reference standard meets the requirements for use as a principle products list of calibrating units for breath alcohol

testers published by the U.S. Department of Transportation in the Federal Register [77 FR 64588 et seq]. * See Dubowski, K.M. and Essary N., Journal of Anal. Toxicol. (1996) 20(6): pp. 484-491. [g/210L = PPM +2605].

Signature

11/14/2022

Name Christopher Pape

Permit No.: 23020



7 Eastgate Dr. • P.O. Box 790 • Jacksonville, IL 62651-0790 217-245-2183 • Fax: 217-243-7634 • www.ilmoproducts.com

Certificate of Analysis

Certificate ID:

15117

Part #:

BAC105L080T

Cylinder Size:

105L

Lot Number:

26422080A1

Expiration:

11/5/2024

MCVV 10/24/22

0.080 BAC (For the calibration of instruments used to determine breath alcohol concentration)

Contents:

105 Liters @ 1000 psig 70°F (21°C)

Analytical

Accuracy

Analytical

Component:

Reported Concentration:

(U, k=2):

Method:

Ethanol

208 ppm

+/-0.002 BAC(G/210L) NDIR

Nitrogen

Balance

[5.2 ppm]

Distributed by:

CMI Inc.

316 East Ninth Street Owensboro, KY 42303 Phone 866-835-0690 www.alcoholtest.com

*Traceable to: NIST Research Gas Mixture - 212.6 µmol/mol Ethanol in Nitrogen - Serial No. SA15944

Store in dry area, away from sources of heat, ignition and direct sunlight. Do not allow storage area to exceed 52 °C (125 °F).

Specialty Gas Lab Tech

10-14-2022

Issuance Date



The calibration results within this certificate were obtained at the facility listed above using equipment and standards capable of producing analytical results traceable to NIST, and apply only to the items contained on this certificate. ILMO Products Company makes no warranty or representation as to the suitability of the use of any information provided for any particular purpose. The information use is at the sole discretion and risk of the user. Liability shall be limited to established replacement cost of this material or service. This certificate applies only to the items described and shall not be reproduced other than in full, without written approval from the issuing facility.

OFFICER'S IMPAIRED DRIVING AFFIDAVIT

Andrew Comments										Section
ARREST DATE 01/05/2023	ARREST TIME 1415		55	CI.	TY#	C	ITATION#			
ARREST LOCATION BOT			CITY					OK	LAHC	MA
BREATHTEST, DELTA					01/1	985	SEX		HEIGHT 51	
3600 MLK		OKLAHO	MA CITY				TATE OK		73:	C-100
B083463042			0/31/20	22	OK	CLASS	3	RESTR	ICTIONS	ENDORSEMENT
VEHICLE MAKE MOCK	MOCK				MOCK		77.73	ATE OK	N CMV	PLACARD N
On the above date, time, and location, the above na	arried person was arrested, and I had i	reasonable grounds to I	believe the person v	vas driv	ing or in actua	al physical	control of	a motor v	ehicle upor	n a public road

highway, street, turnpike, public parking lot, or other public place, or upon a private road, street, alley or lane which provides access to one or more single or multi-family dwelling within this state while under the influence of alcohol and/or other intoxicating substances as prohibited by law. (Describe driving behavior or circumstances):

MOCK SUBJECT TEST

Describe person's condition (odor, actions, etc.):

POST REPAIR MAINTENANCE, INSTALLED NEW CYLINDER

		THE PERSON V	AS INFORMED OF	THE IMPLIED CO	NSENT TEST REQUEST	
BREATH-ALCOHOL ANALYS	IS RECORD, REPORT OF THE F	PERSON NAMED	N SECTION 1 ABOV	E, and LOG OF T	EST AND MAINTENANCE RECORD.	Section 2
EQUIPMENT I-8000 STANDARD		SERIAL I	O 0 5 0 8 0 TARGET VALU	BOARD	OF TESTS	
Dry			0.080		23020	
MANUFACTURER ILMO	t-0. T		26422	080A1	01/05/2023	DATE CYLINDER EXPIRES 11/05/2024
0BSERVATION START TES	1/05/2023				r) deprivation/observation	Fredrick Color
RESULT TABLE Test	g/210L	Time			prior to and continuing est administration.	STHE STATE
Air Blank Subject Test Breath Volume Air Blank Wait Air Blank Subject Test Breath Volume Air Blank	0.00 0.00 0.00	14:56 14:56 14:57 14:59 14:59 15:00	complian	ered said I ce with the edure of the	1907	
Cal Check Air Blank	0.08	15:00 15:01	PAPE, CE BADGE# 0000000 AGENCY BOARD O		ER PERMIT# 0000023020	Test Result BrAC g/210L O.OO

What happens next?

- The Arresting Officer is required to submit the Officer's Impaired Driving Affidavit to the State of Oklahoma.

 Upon receipt and review of the Officer's Impaired Driving Affidavit and any associated test results, if applicable, the State of Oklahoma may revoke your Oklahoma driving privileges. If you hold a commercial driver license or were driving a commercial motor vehicle, the State of Oklahoma may disqualify your privilege to operate a commercial motor vehicle. You will continue to have driving privileges, if otherwise eligible, unless or until an Order of Revocation and/or Disqualification is issued by the State of Oklahoma.

 The State of Oklahoma will send you the Order of Revocation and/or Disqualification to the address you have on file with Service Oklahoma. To confirm or change your address on file, contact
- Service Oklahoma.

The Revocation and/or Disqualification will be effective 55 days from the date of the Order, including 10 days allowed for mailing, 47 O.S. §§2-116, 753, 754.

The Order of Revocation and/or Disqualification will contain important information about your rights to appeal the revocation and/or disqualification of your driving privileges. To appeal the applicable Order of Revocation and/or Disqualification your petition must be filed with the District Court of the County in which you were arrested within 40 days of the date of the order, including 10 days allowed for mailing, 47 O.S. §§2-116, 6-211.

In accordance with Tit	le 12 O.S. Section 426, "I state under	penalty of perj	ury under the laws of Oklahoma	that the fore	going is true and correct."	Section 4
Date:	Place (location when signed):		Signature of arre	esting officer:		
PAPE, CH	RISTOPHER		BOARD OF TES	STS		
BADGE 0000000484	AGENCY ADDRESS INTER-AGENCY			OOODE	4054252460	
OTHER WITNESSES		- PF1				Section :
1 NAME		TITLE	ADDRESS		PHONE	
NAME 2		TITLE	ADDRESS		PHONE	

8/17

OFFICER'S IMPAIRED DRIVING AFFIDAVIT

									Section
ARREST DATE 01/05/2023		ARREST TIME	COUNTY #		CITY#	CITAT	ION#		Section
ARREST LOCATION BOT		4-1-		CITY			COU	NTY KLAHO	OMA
BREATHTEST,	DELTA				OF BIRTH		EX M	HEIGHT 51	2
3600 MLK			OKLAHOMA C	ITY		OF		ZIP COI	111
B083463042			EXPIRATION DA		STATE	D	RES	RICTIONS	ENDORSEMENT
MOCK		MOCK			MOCK		STATE	N	PLACARD N

On the above date, time, and location, the above named person was arrested, and I had reasonable grounds to believe the person was driving or in actual physical control of a motor vehicle upon a public road, highway, street, turnpike, public parking lot, or other public place, or upon a private road, street, alley or lane which provides access to one or more single or multi-family dwelling within this state while under the influence of alcohol and/or other intoxicating substances as prohibited by law. (Describe driving behavior or circumstances):

MOCK SUBJECT TEST

Describe person's condition (odor, actions, etc.):

POST REPAIR MAINTENANCE, INSTALLED NEW CYLINDER

		THE PERSON V	AS INFORMED OF THE I	IMPLIED CONSEN	T TEST REQUEST	
BREATH-ALCOHOL ANALYSIS	RECORD, REPORT OF THE P	ERSON NAMED	N SECTION 1 ABOVE, an	d LOG OF TEST A	ND MAINTENANCE RECORD.	Section 2
I-8000			005080 B	CIALIST OARD OF	manufacture of the second	
Dry			0.080	2	3020	
MANUFACTURER ILMO			26422080A1 DATE CYLINDER INSTALLED 01/05/2023		DATE CYLINDER EXPIRES 11/05/2024	
0BSERVATION START TEST 1430 01	DATE 1/05/2023				privation/observation	
RESULT TABLE Test	g/210L	Time	THE RESERVE OF THE PARTY.		r to and continuing Iministration.	OF THE STATE
Air Blank Subject Test Breath Volume Air Blank Wait Air Blank Subject Test Breath Volume Air Blank	0.00 0.00 2.082 LITERS 0.00 0.00 0.00 2.148 LITERS 0.00	14:56 14:56 14:57 14:59 14:59 15:00	I administere compliance procedu	with the ap	1907	
Cal Check Air Blank	0.08	15:00 15:01	PAPE, CHRI BADGE# 000000048 AGENCY BOARD OF	34 0	MIT# 000023020	Test Result BrAC g/210L 0.00

What happens next?

Section 3

- The Arresting Officer is required to submit the Officer's Impaired Driving Affidavit to the State of Oklahoma.
- Upon receipt and review of the Officer's Impaired Driving Affidavit and any associated test results, if applicable, the State of Oklahoma may revoke your Oklahoma driving privileges. If you hold
 a commercial driver license or were driving a commercial motor vehicle, the State of Oklahoma may disqualify your privilege to operate a commercial motor vehicle. You will continue to have
 driving privileges, if otherwise eligible, unless or until an Order of Revocation and/or Disqualification is issued by the State of Oklahoma.
- The State of Oklahoma will send you the Order of Revocation and/or Disqualification to the address you have on file with Service Oklahoma. To confirm or change your address on file, contact Service Oklahoma.

The Revocation and/or Disqualification will be effective 55 days from the date of the Order, including 10 days allowed for mailing. 47 O.S. §§2-116, 753, 754.

The Order of Revocation and/or Disqualification will contain important information about your rights to appeal the revocation and/or disqualification of your driving privileges. To appeal the applicable Order of Revocation and/or Disqualification your petition must be filed with the District Court of the County in which you were arrested within 40 days of the date of the order, including 10 days allowed for mailing, 47 O.S. §§2-116, 6-211.

In accordance with Title 12 O.S. Section 426, "I state under penalty of perjury under the laws of Oklahoma that the foregoing is true and correct."						
Date:	Place (location when signed):	Signature of arresting officer:				
PAPE,	CHRISTOPHER	BOARD OF TESTS				
BADGE	AGENCY ADDRESS O484 INTER - AGENCY	ZIP CODE				



Dr. Jarrad Wagner, Ph.D., F-ABFT Chairman

Post Office Box 36307 Oklahoma City, Oklahoma 73136-2307 Phone: (405) 425-2460 Fax: (405) 425-2490 www.bot.ok.gov

Joshua Smith Director

Certificate of Analysis - Reference Solution

ALPHA INSTRUMENT

Intoxilyzer - Alcohol Analyzer

Model 8000

SN 80-002591

09/23/2022

09:37

Test	g/210L	Time
Air Blank	0.000	09:37
Cal Check	0.019	09:38
Air Blank	0.000	09:39
Cal Check	0.019	09:39
Air Blank	0.000	09:40
Cal Check	0.019	09:41
Air Blank	0.000	09:41
Cal Check	0.019	09:42
Air Blank	0.000	09:42
Cal Check	0.019	09:43
Air Blank	0.000	09:44
Cal Check	0.019	09:44
Air Blank	0.000	09:45
Cal Check	0.019	09:46
Air Blank	0.000	09:46
Cal Check	0.019	09:47
Air Blank	0.000	09:47
Cal Check	0.019	09:48
Air Blank	0.000	09:49
Cal Check	0.019	09:49
Air Blank	0.000	09:50
Cal Check Stats		
Average	0.0190	
Std Dev	0.0000	
Rel Std Dev(%)	0.0000	

Manufacturer: Guth	Manufactured Date: 8/24/2021	Concentration: 0.02
Lot No.: 21340	Expiration Date: 8/24/2023	Equipment Model: Guth 2100
prescribed by the State Director of Tests pure to 40 OAC 25-1-2.1. Approved breath-alcoho Conforming Products List of Calibrating Units	il measurement equipment and reference solutions. Equ for Breath Alcohol Testers (72 FR 34747), published b	erence solution meets the requirements for use pursuant
Signature // A	Date: 9/23/20	
Name Christopher Pape	Permit No.: 2302	20



Dr. Jarrad Wagner, Ph.D., F-ABFT Chairman

Post Office Box 36307 Oklahoma City, Oklahoma 73136-2307 Phone: (405) 425-2460 Fax: (405) 425-2490 www.bot.ok.gov

Joshua Smith Director

Certificate of Analysis - Reference Solution

ALPHA INSTRUMENT Intoxilyzer - Alcohol Analyzer Model 8000

SN 80-002591

10/24/2022

15:45

Test	g/210L	Time
Air Blank	0.000	15:45
Cal Check	0.039	15:46
Air Blank	0.000	15:47
Cal Check	0.039	15:47
Air Blank	0.000	15:48
Cal Check	0.039	15:49
Air Blank	0.000	15:49
Cal Check	0.039	15:50
Air Blank	0.000	15:50
Cal Check	0.039	15:51
Air Blank	0.000	15:52
Cal Check	0.039	15:52
Air Blank	0.000	15:53
Cal Check	0.039	15:54
Air Blank	0.000	15:54
Cal Check	0.039	15:55
Air Blank	0.000	15:56
Cal Check	0.039	15:56
Air Blank	0.000	15:57
Cal Check	0.040	15:57
Air Blank	0.000	15:58
Cal Check Stats		
Average	0.0391	
Std Dev	0.0003	
Rel Std Dev(%)	0.8087	

Manufacturer: Guth	Manufactured Date: 10/4/2021	Concentration: 0.040	
Lot No.: 21410	Expiration Date: 9/30/2023	Equipment Model: Guth 2100	
to 40 OAC 25-1-2.1. Approved breath-alco Conforming Products List of Calibrating Ur	oursuant to 40 OAC 30-1-3. The above NIST traceable ref ohol measurement equipment and reference solutions. Eq nits for Breath Alcohol Testers (72 FR 34747), published b	lipment used appears on the current or supplemented	
Signature Miles Park	Date: 10/24/2	Date: 10/24/2022	
Name Christopher Pape	Permit No.: 2302	Permit No.: 23020	



Dr. Jarrad Wagner, Ph.D., F-ABFT Chairman

Post Office Box 36307 Oklahoma City, Oklahoma 73136-2307 Phone: (405) 425-2460 Fax: (405) 425-2490 www.bot.ok.gov

Joshua Smith Director

Certificate of Analysis - Reference Solution

ALPHA INSTRUMENT

Intoxilyzer - Alcohol Analyzer

Model 8000

SN 80-002591

06/27/2022

11:40

Test	g/210L	Time
Air Blank	0.000	11:40
Cal Check	0.100	11:41
Air Blank	0.000	11:41
Cal Check	0.100	11:42
Air Blank	0.000	11:43,
Cal Check	0.100	11:43
Air Blank	0.000	11:44
Cal Check	0.100	11:45
Air Blank	0.000	11:45
Cal Check	0.100	11:46
Air Blank	0.000	11:47
Cal Check	0.101	11:47
Air Blank	0.000	11:48
Cal Check	0.101	11:49
Air Blank	0.000	11:49
Cal Check	0.101	11:50
Air Blank	0.000	11:50
Cal Check	0.100	11:51
Air Blank	0.000	11:52
Cal Check	0.100	11:52
Air Blank	0.000	11:53
Cal Check Stats		
Average	0.1003	
Std Dev	0.0004	
Rel Std Dev(%)	0.4815	

Manufacturer: Guth	Manufactured Date: 03	3/07/2022	Concentration: 0.10
Lot No.: 22080	Expiration Date: 03/0	7/2024	Equipment Model: Guth 2100
to 40 OAC 25-1-2.1. Approved breath-a Conforming Products List of Calibrating	ilcohol measurement equipment and re	eference solutions Fau	rence solution meets the requirements for use pursuan ipment used appears on the current or supplemented the National Highway Traffic Safety Administration.
Name Christopher Pape		Date: 06/27/2022	
Name Christopher Pape		Permit No.: 23020	



Dr. Jarrad Wagner, Ph.D., F-ABFT Chairman

Post Office Box 36307 Oklahoma City, Oklahoma 73136-2307 Phone: (405) 425-2460 Fax: (405) 425-2490 www.bot.ok.gov

Joshua Smith Director

Certificate of Analysis - Reference Solution

ALPHA INSTRUMENT

Intoxilyzer - Alcohol Analyzer

Model 8000

SN 80-002591

06/27/2022

11:11

Test	g/210L	Time
Air Blank	0.000	11:11
Cal Check	0.199	11:12
Air Blank	0.000	11:12
Cal Check	0.200	11:13
Air Blank	0.000	11:14
Cal Check	0.200	11:14
Air Blank	0.000	11:15
Cal Check	0.200	11:16
Air Blank	0.000	11:16
Cal Check	0.201	11:17
Air Blank	0.000	11:17
Cal Check	0.200	11:18
Air Blank	0.000	11:19
Cal Check	0.201	11:19
Air Blank	0.000	11:20
Cal Check	0.200	11:21
Air Blank	0.000	11:21
Cal Check	0.200	11:22
Air Blank	0.000	11:23
Cal Check	0.201	11:23
Air Blank	0.000	11:24
Cal Check Stats		
Average	0.2002	
Std Dev	0.0006	
Rel Std Dev(%)	0.3159	

Manufacturer: Guth	Manufactured Date: 02/07/2022	Concentration: 0.20
Lot No.: 22050	Expiration Date: 02/07/2024	Equipment Model: Guth 2100
prescribed by the State Director of Tests pursu to 40 OAC 25-1-2.1. Approved breath-alcohol r Conforming Products List of Calibrating Units for	ared Spectroscopy utilizing the Intoxilyzer 8000 and w ant to 40 OAC 30-1-3. The above NIST traceable ref neasurement equipment and reference solutions. Equ or Breath Alcohol Testers (72 FR 34747), published b	erence solution meets the requirements for use pursuant
Signature	Date: 06/27/2	022
Name Christopher Pape	Permit No.: 2302	20



KCVD 5/9/2022

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 21340 of
Alcohol Reference Solution for Simulator were analyzed by
gas chromatography on August 25, 2021, using a Perkin Elmer Gas
Chromatograph Autosystem XL S/N: 610N9030209, and found to contain
0.0242% (w/vol) ethyl alcohol. The expiration date for this lot
number is August 24, 2023 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.020 g/210L +/- 3%.


The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN05122004 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.

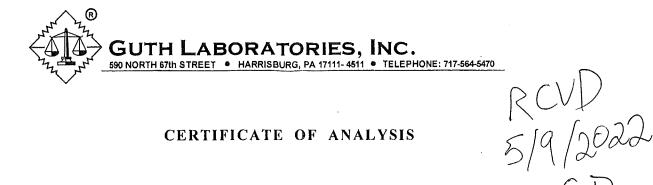

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 21410 Alcohol Reference Solution for Simulator were analyzed by chromatography on October 4, 2021, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.0485% (w/vol) ethyl alcohol. The expiration date for this lot number is September 30, 2023 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.040 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

> Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN08211802 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.



5/9/2022

Certified Alcohol Reference Solution for Simulator

P

Random Samples of Lot Number 22080 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on March 9, 2022, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1214% (w/vol) ethyl alcohol. The expiration date for this lot number is March 7, 2024 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN03052002 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.



Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 22050 of
Alcohol Reference Solution for Simulator were analyzed by
gas chromatography on February 9, 2022, using a Perkin Elmer Gas
Chromatograph Autosystem XL S/N: 610N9030209, and found to contain
0.2437% (w/vol) ethyl alcohol. The expiration date for this lot
number is February 7, 2024 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.200 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN06231703 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.

Location Returned From Unit Serial # Date: Choctaw PD 80-005080 12/20/22 **Inventory Process** Old Cylinder #:_00521080A2-010 ₩nit checked in to BOT ✓ Old cylinder disposed in inventory Unlink Gas Canister from Unit New Unit Checked out Remove old cylinder from unit (After Lab Evaluation) / Old cylinder archived Check in Procedure Notes from the Field Routine Swap Issues in the Field Connect unit to ethernet connection 172.019.004. 140 Download any stored records _ Leaking breath hose. Unit brought in by Cravens Change unit location to Serial number Update Unit Date and time Previous time: 1/5/23 1040 New Time: 1/5/23 Execute a new Forms load on the unit Lab Evaluation Test Chamber Hold Vacuum?: Breath Hose Hold Vacuum?: DVM 3µm 13400 Chamber Temp 47.00 Snap Test ok? Breath Hose Temp __45 RFI Detected? Peripheral Tests Power LED Green (F6) Red (F7) Speaker (F5) Fan (F9) Gas Regulator (F11) Pump (F10) Receipt paper Is regulator close to Atmospheric Sensor Flow Sensor Tank Sensor Value? Correction Factor 1.04 Pressure 968 Reason Sent Sent to CMI (If needed) Date Sent _ Invoice Sent Date Returned Repairs needed: Repairs Made: No issues detected Ready for Bench Check? ☐ Bench Check Technician Report To complete if unit sent to CMI **Inventory Process** Linked Cylinder to Unit ☐ Mock Subject Test Completed ☐ CMI Workorder ☐ Changed Unit Status ☐ CMI Certificate of Calibration ☐ Certificate of Calibration and Operation ☐ Bench Check Report Completed ☐ Service Overview Completed ☐ CMI Service Evaluation Form For BOT Internal Use Only Intoxilyzer Check-in sheet Version 2.0