

STATE OF OKLAHOMA BOARD OF TESTS FOR ALCOHOL AND DRUG INFLUENCE

Dr. Jarrad Wagner, Ph.D. F-ABFT Chairman

Post Office Box 36307 Oklahoma City, Oklahoma 73136-2307 Phone: (405) 425-2460 Fax: (405) 425-2490 www.bot.ok.gov

Joshua Smith Director

ATTESTATION

I, the undersigned <u>Joshua Smith</u> (Director/Records Custodian), attest under oath pursuant to 12 O.S. §2902 (11)(a) this record was made, at or near the time of the occurrence of the matters set forth by or from information transmitted by a person having knowledge of those matters; was kept in the course of the regularly conducted business activity of the Board; and was made pursuant to the regularly conducted activity. The duplicate record provided is listed below:

1) The attached "Service Overview" is a true and accurate copy of maintenance performed on the Intoxilyzer 8000, serial number 80-005081, in compliance with the agency's Maintenance Policy and Procedures.

This packet consists of 13 pages.

Signed ______

Name of Position: Director/Records Custodian

Date of Attestation: 05/24/2022

Service Overview



3600 N. Martin Luther King Bldg #9 Oklahoma City, OK 73111 405-425-2460

BENCH CHECK DATE: <u>02-15-2019</u>

SERIAL: <u>80-005081</u>

DOCUMENTATION WITHIN SERVICE PACKET:

- BOT Certificate of Calibration and Operation
- Bench Check
- Bench Check Technician's Report
- BOT Dry Gas Certificate of Analysis
- Mock Subject Test
- BOT Certificates of Analysis on Certified Solutions (4)



STATE OF OKLAHOMA BOARD OF TESTS FOR ALCOHOL AND DRUG INFLUENCE

Administrative Office at:
Board of Tests for Alcohol and Drug Influence
Post Office Box 36307
Oklahoma City, Oklahoma 73136-2307

Certificate of Calibration and Operation

This is to certify that the calibration of INTOXILYZER 8000, serial number 80-005081, manufactured by CMI Inc., a subsidiary of MPD, Inc. of Owensboro, Kentucky, was tested and found to be in compliance with National Highway Traffic Safety Administration Standard for Devices to Measure Breath Alcohol (F.R., No. 179 48705-48710 Sept. 17, 1993) and with International Recommendation OIML R 126 Edition 1998 (E). Calibration solutions are traceable to NIST standard material 1828.

In addition the operational procedures and functions of this Intoxilyzer have also been tested and found to be in compliance with the Oklahoma Board of Tests for Alcohol and Drug Influence - Rules and Regulations OAC Title 40: 25-1-2 and 30-1-3 and is hereby certified for use as an evidential breath testing device within this state.

OEDTICION TODA	
CERTIFICATION TYPE	DATE
Manufacturer Calibration	05-04-10
Board of Tests Bench Check	06-21-10
Board of Tests Bench Check	05-09-12
Manufacturer Calibration	11-25-13
Board of Tests Bench Check	12-02-13
Board of Tests Bench Check	10-05-15
Board of Tests Bench Check	03-30-17
Board of Tests Bench Check	02-15-19



Presentation of this form certifies that the Intoxilyzer, SN: 80-005081, functioned properly at the time of the breath test and hereby certifies the breath test result as valid.

Date:

Re: Request for Log of Test on Intoxilyzer 8000

Name:

Dear Sir:

The Log of Test information from a breath test conducted on a Intoxilyzer 8000 is printed in Section 2 of the Affidavit and is no longer retained as a separate form as with the Intoxilyzer 5000. The 8000 uses a nitrogen/ethanol dry gas for the external control test. The dry gas cylinder has the capacity to perform approximately 325 tests and does not require maintenance until its expiration date or can no longer provide the required pressure to perform a control test. The cylinder has an expiration date of two years from date filled at the manufacturer and is programmed in the 8000 upon installation, both dates are printed in Section 2 of the Affidavit. The Intoxilyzer is programmed to disable itself when the date of expiration is reached or the pressure monitor indicates 25 lbs psi.

Due to the reliability and stability of dry gas, maintenance on the 8000 is not required every 30 days or 25 tests, therefore maintenance is no longer performed by officers in the field. The Board of Tests for Alcohol and Drug Influence certifies each Intoxilyzer 8000 for field use by performing a bench check. The bench check is designed to check the operation and procedures programmed in the 8000 to ensure it is functioning properly for use in Oklahoma. A bench check is performed every time a new dry gas cylinder is installed or the instrument has returned from the manufacturer for repair. Once the instrument passes the bench check, all that is required to set it up for field use is to plug it in and turn it on and does not require a permit to set up. The Board of Tests has the ability to monitor each instrument once it is set up in the field.

Each Intoxilyzer has a database of valid permit numbers that it checks prior to allowing a breath test to be performed. The database is updated on a regular basis, if a permit number is invalid that officer will not be able to conduct a breath test on the 8000. If the 8000 prints an affidavit the breath test operator listed in Section 2 had a valid permit at the time of the test and the breath test is valid.

If you have any further questions please feel free to call me.

Sincerely.

J. Robert Blakeburn (OBA11059) State Director of Tests for Alcohol and Drug Influence

pohrt Blokelung

STATE OF OKLAHOMA - DEPARTMENT OF PUBLIC SAFETY CMI INTOXILYZER BENCH CHECK PROCEDURE REPORT

02/15/2019	15:02		80-005081				
		ENT MODEL		80-00			
		TEST RE	SULTS	. ~			
Diagnost	ics						
PASS							
Mouth Alcoho	ol Test						
PASS							
RFI Tes	t						
PASS		Dui	olicate Breath	Sampling Re	esults		
Abort Te	st		Jiouto Brouin	Junipining 110	S. CERLEVER		
PASS		Value	Test 1	Test 2	Control Test		
Improper Sam				A . 4 . 5 . 5			
PASS		.02	0.017	0.017	0.017		
Interferent Detect Test		.04	0.037	0.037	0.038		
PASS		.10	0.097	0.097	0.098		
No Sample Giv		.20	0.196	0.195	0.200		
PASS		.20			0.200		
Insufficient Sam	ple Test		PA	SS			
PASS							
Air Blank Contami PASS	nation Test						
Range Exceed	ed Test						
PASS							
		CERTIFYING T	ECHNICIAN				
Lauran undar nanalts	v of monitum, that i			•	1		
I swear under penalty the Intoxliyzer Bencl							
ATURE		27.00		DATE			
				m*	2/15/2019		
				0.	112/2011		
				PERMIT#	10/15/10 V		
MITH, JOSHUA	(-			0000	016837		

3600 N. Martin Luther King Bldg #9 Oklahoma City, OK 73111 405-425-2460

BENCH CHECK TECHNICIAN'S REPORT

SAS CANISTER LOT # EXP Date	Intoxilyzer Model		Serial #	
Section Simulator Model Simulator Model Suth 2100 Guth 210 Serial #	8000		80-00568	
Simulator Model			EVD D-4-	·
Simulator Mode Guth 2100		* 0111		21
Simulator Model Guth 2100 Guth 2100 Guth 2100 Guth 21 Serial # DR3591 DR3753 DR3594 DR375 Concentration .02 .04 .10 .20 LOT # 18020 17410 18070 17340 Manufactured Date 01-09-2018 12-06-2017 02-26-2018 10-09-20 Expiration Date 01-09-2020 12-06-2019 02-26-2020 10-09-20 Solution Commission Date 2-15-19 2-15-19 2-15-19 REASON FOR BENCH CHECK POST REPAIR CYLINDER REPLACEMENT TROUBLESHOOTING ROUTINE MAINTENANCE		- 641	02-09-20	-1
Guth 2100 Guth 2100 Guth 210 Serial # DR3591 DR3753 DR3594 DR375 Concentration .02 .04 .10 .20 LOT # 18020 17410 18070 17340 Manufactured Date 01-09-2018 12-06-2017 02-26-2018 10-09-20 Expiration Date 01-09-2020 12-06-2019 02-26-2020 10-09-20 Solution Commission Date 2-15-19 2-15-19 2-15-19 REASON FOR BENCH CHECK POST REPAIR CYLINDER REPLACEMENT TROUBLESHOOTING ROUTINE MAINTENANCE	KEFEKENCE	Simula	etor Model	
Serial # DR3591 DR3753 DR3594 DR375 Concentration .02 .04 .10 .20 LOT # 18020 17410 18070 17340 Manufactured Date 01-09-2018 12-06-2017 02-26-2018 10-09-20 Expiration Date 01-09-2020 12-06-2019 02-26-2020 10-09-20 Solution Commission Date 2-15-19 2-15-19 2-15-19 REASON FOR BENCH CHECK POST REPAIR CYLINDER REPLACEMENT TROUBLESHOOTING ROUTINE MAINTENANCE	Guth 2100			Guth 2100
Concentration .02 .04 .10 .20 LOT # 18020 17410 18070 17340 Manufactured Date 01-09-2018 12-06-2017 02-26-2018 10-09-20 Expiration Date 01-09-2020 12-06-2019 02-26-2020 10-09-20 Solution Commission Date 2-15-19 2-15-19 2-15-19 REASON FOR BENCH CHECK POST REPAIR CYLINDER REPLACEMENT TROUBLESHOOTING ROUTINE MAINTENANCE				
.02 .04 .10 .20 LOT # 18020 17410 18070 17340 Manufactured Date 01-09-2018 12-06-2017 02-26-2018 10-09-20 Expiration Date 01-09-2020 12-06-2019 02-26-2020 10-09-20 Solution Commission Date 2-15-19 2-15-19 2-15- REASON FOR BENCH CHECK □ POST REPAIR □ CYLINDER REPLACEMENT □ TROUBLESHOOTING □ ROUTINE MAINTENANCE	DR3591	The state of the s		DR3754
LOT # 18020 17410 18070 17340 Manufactured Date 01-09-2018 12-06-2017 02-26-2018 10-09-20 Expiration Date 01-09-2020 12-06-2019 02-26-2020 10-09-20 Solution Commission Date 2-15-19 2-15-19 2-15- REASON FOR BENCH CHECK POST REPAIR □CYLINDER REPLACEMENT □TROUBLESHOOTING □ROUTINE MAINTENANCE				122
18020 17410 18070 17340 Manufactured Date 01-09-2018 12-06-2017 02-26-2018 10-09-200 Expiration Date 01-09-2020 12-06-2019 02-26-2020 10-09-200 Solution Commission Date 2-15-19 2-15-19 2-15-19 REASON FOR BENCH CHECK □POST REPAIR □CYLINDER REPLACEMENT □TROUBLESHOOTING □ROUTINE MAINTENANCE	.02			.20
Manufactured Date 01-09-2018 12-06-2017 02-26-2018 10-09-200 Expiration Date 01-09-2020 12-06-2019 02-26-2020 10-09-200 Solution Commission Date 2-15-19 2-15-19 2-15-19 REASON FOR BENCH CHECK POST REPAIR CYLINDER REPLACEMENT TROUBLESHOOTING ROUTINE MAINTENANCE	18020			17340
01-09-2018 12-06-2017 02-26-2018 10-09-200 Expiration Date 01-09-2020 12-06-2019 02-26-2020 10-09-200 Solution Commission Date 2-15-19 2-15-19 2-15-19 REASON FOR BENCH CHECK □ POST REPAIR □ CYLINDER REPLACEMENT □ TROUBLESHOOTING □ ROUTINE MAINTENANCE	10020	A Promotion of Add Att L. C. C. L.		17040
Expiration Date	01-09-2018			10-09-2017
01-09-2020 12-06-2019 02-26-2020 10-09-20			ation Date	37,77,127,12
2-15-19 2-15-19 2-15-19 2-15-19 2-15-19 2-15-19 2-15-19 2-15-19 2-15-19 2-15-19	01-09-2020	12-06-2019	02-26-2020	10-09-2019
2-15-19 2-15-19 2-15-19 2-15-19 2-15-19 2-15-19 2-15-19 2-15-19 2-15-19 2-15-19		Solution Co	ommission Date	
REASON FOR BENCH CHECK POST REPAIR CYLINDER REPLACEMENT TROUBLESHOOTING ROUTINE MAINTENANCE	2-15-19			2-15-19
□POST REPAIR □CYLINDER REPLACEMENT □TROUBLESHOOTING □ROUTINE MAINTENANCE	- 10 11	2-13-1-1	2-13-11	2 12 1
		EYLINDER REPL		
	TROUBLESHOO			

Printed Name

Average test result



BRT Form 5.0



BOT Technician Name and Signature

In accordance with BOT Policy and Procedure Statement BRT-2.1.0, Maintenance, the above referenced dry gas cylinder is suitable for use as an external reference with the intoxilyzer 8000.

00 • Jacksonville, IL 62651-0790

217-245-2103 · Fax: 217-243-7634 · www.ilmoproducts.com

Certificate of Analysis

Certificate ID:

11617

Part #:

Cylinder#

BAC105L080T

Cylinder Size:

105L

Lot Number:

32518080A1 J

Expiration:

2/5/2021 🗸

0.080 BAC (For the calibration of instruments used to determine breath alcohol concentration)

Contents:

105 Liters @ 1000 psig 70°F (21°C)

Analytical

Reported

Accuracy

Analytical

Component:

Concentration:

(U, k=2):

Method:

Ethanol Nitrogen 208 ppm

+/-0.002 BAC(G/210L) NDIR [5.2 ppm]

Balance 15.

Distributed by:

CMI Inc.

316 East Ninth Stre Owensboro, KY 42: Phone 866-835-069U

www.alcoholtest.com

Operator's Signature

7/13

g/210L

0.000

0.081

0.000

0.081

0.000

0.081

0.000

0.081

0.000

0.081

0.000

0.081

0.000

0.081

0.000

0.082

0.000

0.082

0.000

0.081

0.000

0.0812

0.0004

0.5192

SN 80-004923

09:24

Time

09:24

09:25

09:25

09:25

09:26

09:26

09:27

09:27

09:28

09:28

09:29

09:29

09:29

09:30

09:30

09:31

09:31

09:31

09:32

09:32

09:33

ALPHA INSTRUMENT 2

Model 8000

02/14/2019

Test

Air Blank

Cal Check

Air Blank

Std Deu

Cal Check Stats Average

Rel Std Dev(%)

Intoxilyzer - Alcohol Analyzer

*NIST Traceable Reference Material Cylinder No. CC274523 / Job No. 09160306 Certified 362.2 µmol/mol Ethanol in Nitrogen

Store in dry area, away from sources of heat, ignition and direct sunlight. Do not allow storage area to exceed 52 °C (125 °F).

Specialty Gas Lab Tech

01-62-19

PJLA
Calibration and Testing
Accreditation #61895

The calibration results within this certificate were obtained using equipment and standards capable of producing analytical results traceable to NIST, and apply only to the items contained on this certificate. ILMO Products Company makes no warranty or representation as to the suitability of the use of any information provided for any particular purpose. The information use is at the sole discretion and risk of the user. Liability shall be limited to established replacement cost of this material or service.

OFFICER'S AFFIDAVIT AND NOTICE OF REVOCATION/DISQUALIFICATION

		10	EVOCA		AUYUA			АЦ						Section 1
02/15/20	19		RREST TIME		COUNTY:	#	CIT 9			TATION # VONE				
ARREST LOCATION NONE		1				CITY					COUNT	, FUSK		
SUBJECT NAME							DATE OF			SEX	RACE	HEIGHT		IGHT
MOCK, SU	BJECT	TEST		loi Tr			08/	09/1		M ATE	W	510		.65
3600 MAR'	TIN L	UTHER	KING A	OKI	LAHOMA C	CITY	<i>.</i>		1)K		731		
DRIVER LICENSE #	71				EXPIRATION D		119	STATE	CLASS		RESTRI	CTIONS	ENDOR:	SEMENTS
VEHICLE MAKE	,		MODEL		100/30	720	TAG	; #			ATE	1 1	PLACAF	RD.
MOCK On the above date, time, a	and location.	he above named r	MOCK	I had reasonable o	rounds to believe the	nerson v		OCK	al physical o)K motor ve	N hicle upon	N a public	road
highway, street, turnpike, influence of alcohol and/oi (Describe driving behavior MOCK SUBJE Describe person's condition NONE	r other intoxic or circumsta CT TE	ating substances ances): ST. ROUT	s prohibited by law.	·	·					niiy aweiiir	ng witnin t	nis state wr	mie unde	er the
			THE PERSON V	AS INFORMED OF	F THE IMPLIED CO	NSENT 1	EST REC	DUEST						
BREATH-ALCOHOL ANA	LYSIS RECO	RD, REPORT OF	THE PERSON NAMED	N SECTION 1 ABO	VE, and LOG OF TE				CORD.					Section 2
EQUIPMENT I-8000			SERIAL 80-	NUMBER 005081	SPECIALIST	OF	TES	TS						
STANDARD				TARGET VAL	UE	PERMI	T NUMBE							
Dry MANUFACTURER				0.080)		837	R INSTALL	ED	IDATE	CYLINDE	R EXPIRE	9	
ILMO					3080A1	02	/15	/201	9			/202		
OBSERVATION START 1500	TEST DATE	5/2019	MOUTHPIECE G	A 15-min	ute (or greater) depr	ivatior	ı/observ	ation					
RESULT TABLE	02/1	3/2017		•	was observed p	•			ing			CHE S	FAS	
Test	g/	210L	Time	to	the time of te	st adır	ninistra	ition.			(\$)	***	**	
Air Blank	0.	00	15:51	I adminis	tered said B	reath	ı-Alc	ohol T	est in				***	18
Subject Test		00	15:51	complian	nce with the	appl	icabl	e oper	ating		2 * 1			SE .
Breath Volu Air Blank		503 LITERS	15:52	proc	edure of the	e Boa	ard of	Tests		[집***		**	¥ [8]
Wait			15:53	-						'	(F)*/		U *	
Air Blank Subject Test		00	15:54 15:56											
Breath Volu		761 LITERS		SIGNATURE (OF OPERATOR							1907		
Air Blank Cal Check		00	15:56 15:56	NAME							т	est Re		
Air Blank		08	15:56		JOSHUA	T====						:AC g/2		
				BADGE # 000000	0435	PERMIT		1683	7			-		
				AGENCY	n mpama	L				1	(0.0	0	
FFECTIVE THIRT	V (20) DA	VS EDOM DA	TE SERVED. Not		F TESTS	t	drivorlo	liaanaa	ia ravalta	d o 1 do 1	ind for		,	Section 3
eriod of one hundre his state will be disc commercial motor ve an Administrative He erson at 3600 N. M	ed eighty (qualified if ehicle. To earing witl LK Ave. o	180) days or n you refused of appeal the dri nin fifteen (15 r by mail to Le	nore, if you refused r failed the state's t ver's license revoc) days from the d gal Division, Depa	or failed the si est(s) (operatir ation/disqualifi ate you receiv tment of Public	tate's test(s). In ng a CMV or CD cation, the Depa red notice of re c Safety, P.O. Bo	addition L holder I hold	on, you er) OR t of Put on or o 15, OK	r comme a test re blic Safet disqualif C, OK, 7	ercial driving sult of .04 by must refication. 73136	ng privi 4 or mor ceive y Submit	lege in e while our writ your w i	operatin ten reque ritten red	ng a est for quest i	
SERVING OF NOTE					aining the Notice	e of Re	vocatio	n to the	person n	amed in	Section	n 1 abov	e.	
ate served: 02/	15/	2019 s	ignature of officer:							Bad	ge #:			
accordance with T	itle 12 O.	S. Section 426	, "I state under pe i	nalty of perjur	y under the laws	of Ok	lahoma	that the	foregoin	g is true	and co	rrect."	S	ection 4
ate: 02/15/2	2 <u>019</u> F	Place (location	when signed):			Sigr	nature c	of arrestin	ng officer			Million		
SMITH,	JOSHU.	A			AGENCY BOARD	OF	TE:	STS						
ADGE 000000043	AGEN	ICY ADDRESS ITER-AG	ENCY					ZIP CODE) O PHO		1252	460"	I	
THER WITNESSE	S:												S	ection 5
NAME		· · · · · · · · · · · · · · · · · · ·		TITLE	ADDRESS					PHONE				
1 NAME		***************************************		TITLE	ADDRESS					PHONE				
0				1	i					1				

9/13

OFFICER'S AFFIDAVIT AND NOTICE OF REVOCATION/DISOUALIFICATION

•		TUDIS	CILL	411 .						Section
ARREST DATE	ARREST TIME		COUNTY#		CITY#		CITATION #			
02/15/2019	1450		54		90		NONE			
ARREST LOCATION				CITY				COUNT	Y	
NONE				NO:	NE			OK:	FUSK	${ m EE}$
SUBJECT NAME					TE OF BIRTH		SEX	RACE	HEIGHT	WEIGHT
MOCK, SUBJECT TEST				(08/09/	1961	M	W	510	165
ADDRESS		CITY				[5	STATE		ZIP CODE	
3600 MARTIN LUTHER	KING A	OKLAHO	MA CI	TY			OK		731	33
DRIVER LICENSE #			IRATION DAT		STATE	CLAS	S	RESTRI	CTIONS	ENDORSEMENTS
T083454871		0	6/30/	201	9 OK	A				M
VEHICLE MAKE	MODEL				TAG#		STA	TE	CMV	PLACARD
MOCK	MOCK				MOCK		0	K	N	N
On the above date, time, and location, the above name highway, street, turnpike, public parking lot, or other pul influence of alcohol and/or other intoxicating substance (Describe driving behavior or circumstances):	olic place, or upon a private road, str									
MOCK SUBJECT TEST. ROU	TINE MAINTENANC	E. NEW G	AS CAN	ISTE	ER INST	ALL.				
Describe person's condition (odor, actions, etc.):										

THE REPORT WAS INDODUCED OF THE LAND OF GOVERNMENT THOU PROTECTION

	THE PERSON	VAS INFORMED OF THE IMPLIED CONSENT TEST REQUES	T	
BREATH-ALCOHOL ANALYSIS RECORD, REPORT OF T	HE PERSON NAMED	IN SECTION 1 ABOVE, and LOG OF TEST AND MAINTENANG	CE RECORD.	Section 2
EQUIPMENT	1	NUMBER SPECIALIST		
I-8000	80-	005081 BOARD OF TESTS	\$	
STANDARD		TARGET VALUE PERMIT NUMBER		
Dry		0.080 16837		
MANUFACTURER		LOT# DATE CYLINDER INS		DATE CYLINDER EXPIRES
ILMO		32518080A1 02/15/2	019	02/05/2021
OBSERVATION START TEST DATE 1500 02/15/2019	MOUTHPIECE	A 15-minute (or greater) deprivation/ob	servation	
1500 02/15/2019 RESULT TABLE	G	period was observed prior to and cont		
RESULT TABLE		to the time of test administration		THE STATE
Test g/210L	Time	to the time of test administration	1.	Ø****\^*.*\o\
		Tadoutotakanad satid Dosaki. Alic ib	100	\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\\
Air Blank 0.00	15:51	I administered said Breath-Alcoho	or rest in	B CANDA S IN
Subject Test 0.00	15:51	compliance with the applicable or	perating	
Breath Volume 1.503 LITERS		procedure of the Board of Te		到 ***
Air Blank 0.00	15:52	procedure of the Board of Te	818.	F **
Wait	15:53			
Air Blank 0.00	15:54			
Subject Test 0.00 Breath Volume 1.761 LITERS	15:56	SIGNATURE OF OPERATOR		1907
	15.56			
Air Blank 0.00 Cal Check 0.08	15:56 15:56	NAME		T . (D . 1)
Air Blank 0.00	15:56	SMITH, JOSHUA		Test Result
All Blank 0.00	15:57	BADGE # PERMIT #		BrAC g/210L
		0000000435 00000168	837	
		AGENCY		0.00
		BOARD OF TESTS		

EFFECTIVE THIRTY (30) DAYS FROM DATE SERVED. Notice of Revocation is given that your driver's license is revoked or denied for a period of one hundred eighty (180) days or more, if you refused or failed the state's test(s). In addition, your commercial driving privilege in this state will be disqualified if you refused or failed the state's test(s) (operating a CMV or CDL holder) OR a test result of .04 or more while operating a commercial motor vehicle. To appeal the driver's license revocation/disqualification, the Department of Public Safety must receive your written request for an Administrative Hearing within fifteen (15) days from the date you received notice of revocation or disqualification. Submit your written request in person at 3600 N. MLK Ave. or by mail to Legal Division, Department of Public Safety, P.O. Box 11415, OKC, OK, 73136

SERVING OF NOTICE: I personally hand-delivered a copy of the above containing the Notice of Revocation to the person named in Section 1 above.

Date served: 02/15/2019 Signature of officer:	Badge #:
In accordance with Title 12 O.S. Section 426, "I state under penalty of perjury u	inder the laws of Oklahoma that the foregoing is true and correct." Section 4
Date: 02/15/2019 Place (location when signed):	Signature of arresting officer:
NAME SMITH, JOSHUA	AGENCY DOADD OF HEROEG
•	BOARD OF TESTS
BADGE AGENCY ADDRESS TAITHUR A GRANGY	ZIP CODE PHONE
000000435 INTER-AGENCY	00000 "4054252460"

NONE



Certificate of Analysis – Wet Bath

09-26-2018 , 82 9 0 0 13

Date of Analysis Labelled target value (g/210L)

18829 0.82 94

4511 9 TELEPHONE: 717-564-5470

BOT Technician Name and Signature

BRT Form 6.0

In accordance with BRT – 2.1.0, Maintenance, the above referenced simulator solution is suitable for use as an external reference in maintenance of the Intoxilyzer 8000.

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 18020 of

Alcohol Reference Solution for Simulator were analyzed by
gas chromatography on January 11, 2018, using a Perkin Elmer Gas
Chromatograph Autosystem XL S/N: 610N9030209, and found to contain
0.0238% (w/vol) ethyl alcohol. The expiration date for this lot
number is January 9, 2020 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.020 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

3

Signature

ALPHA INSTRUMENT

Model 8000

09/26/2018

Test

Air Blank

Cal Check

Air Blank

Cal Check

Air Blank Cal Check

Air Blank

Cal Check

Air Blank

Cal Check

Air Blank

Cal Check

Pir Blank

Cal Check

ir Blank

al Check

lir Blank

al Check

ir Blank

al Check

ir Blank

Std Deu

al Check Stats average

Rel Std Deu(%)

Operate

Intoxilyzer - Alcohol Analyzer

g/210L

.0.000

0.020

0.000

0.020

0.000

0.020

0.000

0.020

0.000

0.020

0.000

0.020

0.000

0.021

0.000

0.021

0.000

0.021

0.000

0.021

0.000

0.0204

0.0005

2.5312

SN 80-002591

15:36

Time

15:36

15:37

15:37

15:38 15:39

15:39

15:40

15:41

15:41

15:42

15:42

15:43

15:44

15:44

15:45

15:46

15:46

15:47

15:47

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN08031603 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.

590 NORTH 67th STREET ARRISBURG, PA 17 149t Number: TELEPHONE: 147 564 5470

Certificate of Analysis - Wet Bath 09-26-18 Labelled target value (g/210L) Date of Analysis 0.0396

ANALLYSIS external reference in maintenance of the intoxilyzer 8000.

In accordance with BRT - 2.1.0, Maintenance, the above referenced simulator solution is



CERTIFICATE OF

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number Alcohol Reference Solution for Simulator were analyzed by chromatography on December 7, 2017, using a Perkin Elmer Gas gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.0481% (w/vol) ethyl alcohol. The expiration date for this lot number is December 6, 2019 at 11:59 PM.

BOT Technician Name and Signature

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.040 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Operator Signature

ALPHR INSTRUMENT Intoxilyzer - Alcohol Analyzei

SN 80-002591

15:54

Tine

15:54

15:55

15:56

15:56

15:57

15:58

15:58

15:59

15:59

16:00

16:01

16:02

16:03

16:03

16:04

16:04

16:05

16:06

16:06

16:07

Model 8000

09/26/2018

Test

Air Blank

Cal Check

Rir Blank

Cal Check

Air Blank

Cal Check

Air Blank

Cal Check

Air Blank

Cal Check

Air Blank

Average Std Deu

Cal Check Stats

Rel Std Deu(%)

0.000

0.040

0.000

0.039

0,000

0.039

0.000

0.040

0.000

0.040

0.000

0.040

0.000

0.039

0.000

0.040

0.000

0.040

0.000

0.039

0.000

0.0396


0.0005

L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN12181501 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights, Calibration verification is done prior to each use utilizing NIST traceable weights.

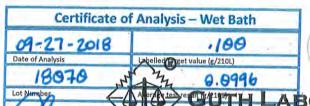

BOT Technician Name and Signature

BLPHA INSTRUMENT

Model 8000

09/27/2018

Test

Air Blank

Cal Check

Air Blank

Ca! Check

Air Blank

Cal Check

Air Blank

Cal Check

Air Blank

Cal Check

Rir Blank

Cal Check

Air Blank

Cal Check

Air Blank

Cal Check

Air Blank

Cal Check

Air Blank

Cal: Check

Air Blank

Std Dev Rel Std Dev(%)

Cal Check Stats Average

Intoxilyzer - Alcohol Analyzer

g/210L

0.000

0.100

0.000

0.099

0.000

0.100

0.000

0.100

0.000

0.100

0.000

0.099

0.000

0.099

0.000

0.099

0.000

0.100

0,000

0.0005

0.5184

SN 80-002591

08:28

Tine

08:28

08:29

08:29

08:30

08:31

08:31

08:32

08:33

08:33

08:34

08:34

08:35

08:36

08:36

08:37

08:38

08:38

08:39

08:39

08:40

08:41

OUTH LABORATORIES, INC.
590 NORTH 67th STREET • HARRISBURG, PA 17111-4511 • TELEPHONE: 717-564-5470

BRT Form 6.0

In accordance with BRT – 2.1.0, Maintenance, the above referenced simulator solution is suitable for use as an external reference in maintenance of the intoxityzer 8000.

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 18070 of

Alcohol Reference Solution for Simulator were analyzed by
gas chromatography on February 28, 2018, using a Perkin Elmer Gas
Chromatograph Autosystem XL S/N: 610N9030209, and found to contain
0.1219% (w/vol) ethyl alcohol. The expiration date for this lot
number is February 26, 2020 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

16837


Operator's Signature

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN04271602 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.

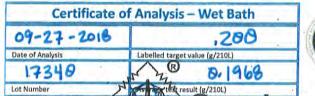

BOT Technician Name and Signature

ALPHA INSTRUMENT

Model 8000

09/27/2018

Air Blank

Cal Check

Rir Blank

Cal Check

Air Blank

al Check

Rir Blank

'al Check

Hir Blank

al Check

Iir Blank

Average Std Dev

al Check Stats

Rel Std Dev(%)

Intoxilyzer - Alcohol Analyzer

g/210L

0.000

0.196

0.000

0.196

0.000

0.197

0.000

0.197

0.000

0.197

0.000

0.197

0.197

0.000

0.197

0.000

0.197

0.000

0.197

0.000

0.1968

0.0004

0,2142

SN 80-002591

15:27

Time

15:27

15:28

15:29

15:29

15:30

15:31

15:31

15:32

15:33

15:33

15:34

15:35

15:36

15:36

15:37

15:38

15:38

15:39

15:40

GUTH LABORATORIES, INC.

HARRISBURG, PA 17111-4511
 TELEPHONE: 717-564-5470

BRT Form 6.0

In accordance with BRT -2.1.0, Mainlenance, the above referenced simulator solution is suitable for use as an external reference in maintenance of the intoxilyzer 8000.

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 17340 of
Alcohol Reference Solution for Simulator were analyzed by
gas chromatography on October 10, 2017, using a Perkin Elmer Gas
Chromatograph Autosystem XL S/N: 610N9030209, and found to contain
0.2410% (w/vol) ethyl alcohol. The expiration date for this lot
number is October 9, 2019 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.200 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Operator's Signature

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN08101505 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.